

INSTRUCTIONS OF USE

One pack of *Corrective Salopette* contains: 1 pair of Salopette + 1 pocket + 2 orthopaedic panels in two sizes


Phase 1: HOW TO PREPARE THE SALOPETTE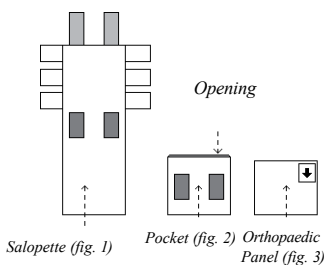

Open the Salopette (fig. 1), with the pocket (fig. 2) and orthopaedic panel (fig. 3) close to hand.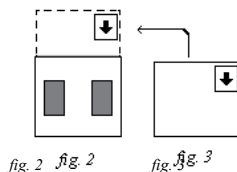

Using the size indicated by the doctor, insert the orthopaedic panel (fig. 3) into the opening of the pocket in the direction indicated by the arrow on the sticker attached to the orthopaedic panel (fig. 2).

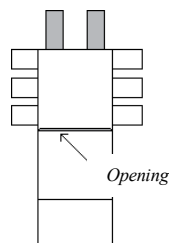Position the pocket containing the orthopaedic panel over the Salopette, lining up the stripes, orienting the opening of the pocket upwards.

Phase 2: HOW TO WEAR THE SALOPETTE

The *Baby System Corrective Salopettes* are designed to be worn over traditional disposable nappies.

- 1) Place the child in a supine position (face up) on top of the Salopette.
- 2) Spread the child's thighs so that the remaining part of the Salopette can be folded over the chest.
- 3) Fasten the side Velcro straps to ensure that the Salopette fits snugly against the torso.
- 4) Set the tension of the cross-straps.

The spread provided by the Salopette is necessary for infants from day 1 when the joint is still forming. It is recommended not to over-use excessively elasticated clothing or baby seats that compress the space around the hips. Machine-wash the BABY SYSTEM Corrective Salopette at 40° after removing the orthopaedic panel and fastening the stramps. Ironing not recommended.

For more information, please contact our customer service or consult your doctor.

Importer:
Helvetica Health Care Sàrl
Voie-de-traz 20
1218 Le Grand-Saconnex (GE) - Switzerland

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