INSTRUCTIONS OF USE

The BABY SYSTEM Corrective Nappies offer physiological support for developing hips. Healthy hip positioning, known as the M position, is essential for infants from day 1 when the joint is still forming. Anything that could bring the legs together (such as the use of overly tight clothes, baby seats, etc.) should be avoided during treatment.

The BABY SYSTEM Corrective Nappies are designed to be worn over traditional disposable nappies and over clothing or baby grows.

The BABY SYSTEM Corrective Nappies are available in three sizes. To avoid injury, please use the size determined by your doctor based on the required hip spread (e.g. 14, 16 or 18 cm).

The BABY SYSTEM Corrective Nappies are made from quick-drying spun-bond fabric and are both machine and hand washable at a maximum temperature of 40°C. Ironing not recommended.

The orthopaedic panel (C) is inserted into the internal pocket (B) as shown in figure 1, with the wider site at the top and the narrower side at the bottom.

Two Velcro straps are sewn into the upper inner part of the Nappies (figure 1), which are used to fasten the splint and adjust the fit by joining them to the Velcro strap (E) sewn horizontally in the lower outer part (figure 2).


The Nappies also feature two adjustable cross-straps (F+G) which provide support when fastening the splint to ensure that the abduction remains effective and the hips are held in the correct position.

The fabric is mite-proof, antibacterial and hypoallergenic. Please note that the colour of the print is non-toxic and not very strong and consequently may fade with washing.

For more information, please contact our customer service or consult your doctor.


Patent N. 00002684565

Fig. 1 - Inner Part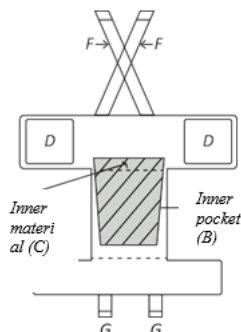

The orthopaedic panel is inserted into the inner pocket of the Corrective Nappies.

Fig. 2 - Outer Part

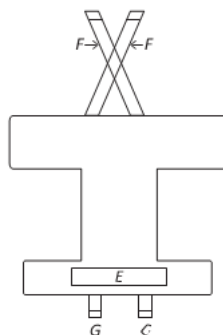The Corrective Nappies are designed to be worn over disposable nappies.

Importer:
Helvetica Health Care Srl
Voie-de-traz 20
1218 Le Grand-Saconnex (GE) -
Switzerland

Distributor:
Povatex s.a.s
Via Alessandro Manzoni, 58
73056 Taurisano (LE) - Italy

Manufacturer:
Povatex s.a.s
Via Alessandro Manzoni, 58
73056 Taurisano (LE) - Italy

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