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(Graf-Methode Typ IIa(-), IIb und IIc)

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IDEAL DYNAMIC HIP HARNESS

(Graf method type IIa(-), IIb and IIc)

A newborn's hips must be carefully treated immediately from birth. By acting preventively, we can avoid a worsening of the newborn's hips and the need for drastic treatments. The DYNAMIC HIP HARNESS was designed for type II hip dysplasia (Graf type classification) and is suitable for use from birth.

It is made from soft but effective materials. The different settings mean it can be worn in various ways; the leg loops can be crossed or uncrossed where needed, based on the treatment, therefore offering a wide range of possibilities to adjust the hip position. These are indicated on the harness itself to facilitate changing for parents.

The lining can be washed up to 50 °C. The internal material must be removed before washing. The harness is made from hypoallergenic materials: cotton exterior and soft expandable interior.

Recommended use:

- size 0 for children from birth to 6 weeks, approximately 45-52 cm (with the child in clothing, measure the chest circumference taking care to pass below the armpits).
- size 1 for children from 6 to 9 weeks, approximately 52-58 cm (with the child in clothing, measure the chest circumference taking care to pass below the armpits).
- size 2 for children from around 9 weeks, approximately 55-62 cm (with the child in clothing, measure the chest circumference taking care to pass below the armpits).

For more information contact our customer service team or speak to your doctor.

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