



Baby System

Panetto morbido

Baby Giò

HOW TO USE

The Baby GIÒ Soft Nappy is suitable for newborns from day 1 to promote healthy physiological posture of the hips while the joints are still forming. It is recommended not to over-use excessively elasticated clothing or baby seats that compress the space around the hips.

The Baby GIÒ Soft Nappy is worn over traditional disposable nappies and can be worn over or under clothing.

The Baby GIÒ Soft Nappy is recommended for use all day and is also suitable for night-time wear.

The Baby GIÒ Soft Nappy features adjustable straps to meet the specific needs of each child.

The Baby GIÒ Soft Nappy is made from quick-drying spun-bond fabric and is both machine and hand washable at a maximum temperature of 40°C. Ironing not recommended.

The fabric is mite-proof, antibacterial and hypoallergenic. Please note that the color of the print is non-toxic and not very strong and consequently may fade with washing.

For more information, contact our customer service or consult your doctor.

Prodotto brevettato
N. 0000229978



Manufacturer:
Povatex s.a.s
Via Alessandro Manzoni, 58
73056 Taurisano (LE) – Italy

