



HBV Longitudinal Panel Part Number: HBV6508

*These reagents are not a substitute for the mandatory positive and negative control reagents provided with licensed test kits.
For Research Use Only. Not for use in diagnostic procedures.*

PANEL NUMBER	DATE OF DRAW	ORTHO VITROS ECi ANTI-HIV ½	ORTHO VITROS ECi ANTI-HCV	ABBOTT HBsAg	ORTHO VITROS ECi HBsAg	ABBOTT ANTI-HBc TOTAL	ORTHO VITROS ECi ANTI-HBc TOTAL	ABBOTT ANTI-HBc IgM
		TEST DATE: 8/4/2014	TEST DATE: 8/4/2014	TEST DATE: UNK	TEST DATE: 8/4/2014	TEST DATE: UNK	TEST DATE: 8/4/2014	TEST DATE: UNK
6508-01	05/16/01	0.04	0.08	>40.8	1480	0.04	0.08	>6.9
6508-02	05/28/01	0.09	0.07	>40.8	2790	0.05	0.10	>6.9
6508-03	06/11/01	0.09	0.06	>40.8	3100	0.04	0.09	>6.9
6508-04	06/28/01	0.10	0.03	>40.8	6760	0.05	0.05	>6.9
6508-05	07/11/01	0.06	0.03	>40.8	6670	0.06	0.46	>6.9
6508-06	07/25/01	0.09	0.05	7.1	0.60	0.04	0.15	3.1
6508-07	08/08/01	0.08	0.04	0.08	0.07	0.03	0.10	3.0
6508-08	08/22/01	0.07	0.05	0.04	0.07	0.03	0.07	3.4
6508-09	09/05/01	0.08	0.05	0.09	0.08	0.04	0.07	2.9
6508-10	09/19/01	0.06	0.05	0.09	0.07	0.03	0.06	2.9
6508-11	10/03/01	0.08	0.05	0.09	0.06	0.04	0.05	2.9
6508-12	10/16/01	0.06	0.05	0.04	0.07	0.07	0.05	2.5
6508-13	11/20/01	0.06	0.02	0.08	0.08	0.08	0.03	2.2
6508-14	12/11/01	0.06	0.03	0.02	0.09	0.08	0.02	2.1
6508-15	12/25/01	0.03	0.04	0.13	0.08	0.07	0.05	2.2

PANEL NUMBER	ORTHO VITROS ECi ANTI-HBc IgM	DIASORIN ANTI-HBe	ORTHO VITROS ECi ANTI-HBe	ABBOTT ANTI-HBs QUANT	ORTHO VITROS ECi ANTI-HBs QUANT	ORTHO VITROS ECi HBeAg	DIASORIN HBeAg
	TEST DATE: 8/4/2014	TEST DATE: UNK	TEST DATE: 8/4/2014	TEST DATE: UNK	TEST DATE: 8/4/2014	TEST DATE: 8/4/2014	TEST DATE: UNK
6508-01	QNS	0.6	0.65	0.3	QNS	0.60	0.3
6508-02	3.84	0.9	0.28	0.3	1.8	0.39	0.3
6508-03	3.91	0.9	0.33	0.3	1.7	0.37	0.3
6508-04	3.88	1.0	0.62	0.2	1.5	0.45	0.3
6508-05	3.40	0.6	QNS	0.3	1.3	0.13	0.5
6508-06	1.83	0.1	19.2	0.4	3.8	0.11	0.07
6508-07	4.41	0.1	33.4	1.2	9.8	0.11	0.37
6508-08	1.58	0.1	16.9	1.95	19.2	0.11	0.07
6508-09	1.64	0.1	17.9	4.3	54.3	0.11	0.12
6508-10	1.50	0.1	10.4	7.5	116	0.11	0.08
6508-11	1.37	0.2	12.8	10.4	150	0.12	0.14
6508-12	1.33	0.2	8.15	9.3	210	0.11	0.04
6508-13	1.23	0.2	4.43	13.0	232	0.11	0.12
6508-14	0.95	0.3	2.72	10.9	184	0.11	0.08
6508-15	1.21	0.4	2.70	11.6	228	0.10	0.12

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
 This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
 For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com