

# ZeptoMetrix®

## HBV Longitudinal Panel Part Number: HBV6511

*These reagents are not a substitute for the mandatory positive and negative control reagents provided with licensed test kits.  
For Research Use Only. Not for use in diagnostic procedures.*

**\*Donor is co-infected with Hepatitis C.**

PANEL NUMBER	DATE OF DRAW	ORTHO VITROS ECi ANTI-HIV ½	ORTHO VITROS ECi ANTI-HCV	ABBOTT HBsAg	ORTHO VITROS ECi HBsAg	ABBOTT ANTI-HBc TOTAL	ORTHO VITROS ECi ANTI-HBc TOTAL	ABBOTT ANTI-HBc IgM
		TEST DATE: 8/8/2014	TEST DATE: 8/8/2014	TEST DATE: UNK	TEST DATE: 8/8/2014	TEST DATE: UNK	TEST DATE: 8/8/2014	TEST DATE: UNK
6511-01	05/30/01	0.05	33.3	>40.8	3380	0.05	0.03	>6.9
6511-02	06/13/01	0.06	31.5	>40.8	7500	0.26	0.08	>6.9
6511-03	06/28/01	0.07	32.2	>40.8	8610	0.13	0.04	>6.9
6511-04	07/11/01	0.07	32.4	>40.8	7080	0.11	0.03	>6.9
6511-05	07/25/01	0.08	32.6	>38	3060	0.03	0.02	4.5
6511-06	08/08/01	0.11	32.6	>38	116	0.04	0.02	3.0
6511-07	08/22/01	0.07	32.1	8.8	1.93	0.04	0.02	2.5
6511-08	09/05/01	0.06	32.0	0.05	0.06	0.04	0.02	2.0
6511-09	09/19/01	0.07	31.1	0.09	0.18	0.04	0.02	1.6
6511-10	10/03/01	0.06	31.2	0.08	0.06	0.03	0.02	1.6
6511-11	10/17/01	0.07	30.8	0.05	0.05	0.05	0.02	1.5
6511-12	10/30/01	0.09	30.0	0.05	0.06	0.05	0.03	1.3

PANEL NUMBER	ORTHO VITROS ECi ANTI-HBc IgM	DIASORIN ANTI-HBe	ORTHO VITROS ECi ANTI-HBe	ABBOTT ANTI-HBs QUANT	ORTHO VITROS ECi ANTI-HBs QUANT	ORTHO VITROS ECi HBeAg	DIASORIN HBeAg
	TEST DATE: 8/8/2014	TEST DATE: UNK	TEST DATE: 8/8/2014	TEST DATE: UNK	TEST DATE: 8/8/2014	TEST DATE: 8/8/2014	TEST DATE: UNK
6511-01	3.87	0.2	3.01	0.3	2.4	0.26	0.2
6511-02	3.84	0.2	2.33	0.2	2.7	0.12	0.2
6511-03	3.53	0.1	10.0	0.3	1.5	0.16	0.2
6511-04	2.88	0.5	23.7	0.2	1.7	0.20	0.2
6511-05	2.09	0.01	36.5	0.27	0.7	0.12	0.01
6511-06	1.35	0.02	41.3	0.37	0.2	0.23	0.01
6511-07	1.19	0.01	34.6	0.31	2.6	0.11	0.01
6511-08	0.98	0.01	32.2	0.6	4.7	0.13	0.01
6511-09	0.83	0.01	8.98	0.81	10.4	0.38	0.03
6511-10	0.69	0.01	28.0	2.0	23.3	0.12	0.01
6511-11	0.66	0.01	28.9	3.4	47.6	0.11	0.03
6511-12	0.55	0.01	25.8	7.3	70.6	0.10	0.01

### Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)

PIHBV6511 Rev03  
Effective Date: 09/01/2021