



**HBV Longitudinal Panel  
Part Number: HBV6538**

*These reagents are not a substitute for the mandatory positive and negative control reagents provided with licensed test kits.  
For Research Use Only. Not for use in diagnostic procedures.*

**\*Donor is co-infected with Hepatitis C.**

PANEL NUMBER	DATE OF DRAW	ORTHO VITROS ECi ANTI-HIV ½	ORTHO VITROS ECi ANTI-HCV	ABBOTT HBsAg	ORTHO VITROS ECi HBsAg	ABBOTT ANTI-HBc TOTAL	ORTHO VITROS ECi ANTI-HBc TOTAL	ABBOTT ANTI-HBc IgM
		TEST DATE: 8/21/2014	TEST DATE: 8/21/2014	TEST DATE: UNK	TEST DATE: 8/21/2014	TEST DATE: UNK	TEST DATE: 8/21/2014	TEST DATE: UNK
6538-01	11/28/01	0.10	35.1	>37	3700	0.04	0.02	>6.5
6538-02	12/12/01	0.13	35.2	>37	7910	0.04	0.02	5.2
6538-03	12/26/01	0.12	36.3	>37	9150	0.1	0.02	4.2
6538-04	01/08/02	0.11	36.2	>37	6660	0.1	0.02	3.1
6538-05	01/22/02	0.12	34.8	>37	2160	0.1	0.02	2.4
6538-06	02/06/02	0.09	34.2	>37	96.1	0.03	0.04	1.6
6538-07	02/19/02	0.10	33.3	>37	166	0.1	0.02	2.1
6538-08	03/05/02	0.10	33.0	2.4	0.70	0.1	0.02	1.9
6538-09	03/19/02	0.10	31.9	1.1	0.09	0.1	0.02	1.6
6538-10	04/02/02	0.13	33.2	0.2	0.10	0.1	0.03	1.9
6538-11	04/16/02	0.09	32.6	0.2	0.08	0.1	0.03	1.5
6538-12	04/30/02	0.10	33.2	0.2	0.07	0.1	0.02	1.5

PANEL NUMBER	ORTHO VITROS ECi ANTI-HBc IgM	DIASORIN ANTI-HBe	ORTHO VITROS ECi ANTI-HBe	ABBOTT ANTI-HBs QUANT	ORTHO VITROS ECi ANTI-HBs QUANT	ORTHO VITRO ECi HBeAg	DIASORIN HBeAg
	TEST DATE: 8/21/2014	TEST DATE: UNK	TEST DATE: 8/21/2014	TEST DATE: UNK	TEST DATE: 8/21/2014	TEST DATE: 8/21/2014	TEST DATE: UNK
6538-01	2.62	0.04	23.0	0.3	2.3	0.19	0.08
6538-02	1.95	0.05	24.3	0.3	0.6	0.16	0.12
6538-03	1.58	0.05	19.6	0.3	2.0	0.18	0.04
6538-04	1.22	0.04	20.2	0.3	0.5	0.17	0.12
6538-05	1.04	0.05	24.8	0.4	1.2	0.17	0.24
6538-06	0.86	0.05	33.2	0.2	3.2	0.12	0.12
6538-07	0.99	0.03	25.9	0.3	2.5	0.16	0.04
6538-08	0.74	0.05	27.7	0.3	0.8	0.16	0.14
6538-09	0.65	0.04	22.7	0.2	1.1	0.15	0.07
6538-10	0.64	0.05	28.5	0.3	1.2	0.16	0.18
6538-11	0.58	0.05	23.0	0.4	2.6	0.17	0.08
6538-12	0.60	0.04	21.5	0.4	2.8	0.15	0.12

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)