

# ZeptoMetrix®

## HBV Longitudinal Panel Part Number: HBV6542

*These reagents are not a substitute for the mandatory positive and negative control reagents provided with licensed test kits.  
For Research Use Only. Not for use in diagnostic procedures .*

**\*Donor is co-infected with Hepatitis C.**

PANEL NUMBER	DATE OF DRAW	ORTHO VITROS ECi ANTI-HIV ½	ORTHO VITROS ECi ANTI-HCV	ABBOTT HBsAg	ORTHO VITROS ECi HBsAg	ABBOTT ANTI-HBc TOTAL	ORTHO VITROS ECi ANTI-HBc TOTAL	ABBOTT ANTI-HBc IgM
		TEST DATE: 8/21/2014	TEST DATE: 8/21/2014	TEST DATE: UNK	TEST DATE: 8/21/2014	TEST DATE: UNK	TEST DATE: 8/21/2014	TEST DATE: UNK
6542-01	12/25/01	0.14	26.3	>37	6200	0.04	0.18	>6.5
6542-02	01/08/02	0.12	25.1	>37	7730	0.1	0.16	>6.5
6542-03	01/22/02	0.14	24.0	>37	457	0.1	0.20	4.9
6542-04	02/05/02	0.12	22.5	2.9	1.00	0.1	0.15	3.4
6542-05	02/19/02	0.13	21.4	0.3	0.06	0.1	0.08	2.4
6542-06	03/05/02	0.12	20.5	0.1	0.05	0.1	0.07	1.8
6542-07	03/19/02	0.11	17.5	2.1	0.06	0.1	0.04	1.4
6542-08	04/02/02	0.15	26.3	0.1	0.06	0.04	0.05	1.2
6542-09	04/16/02	0.12	29.7	0.1	0.05	0.1	0.04	1.1
6542-10	04/30/02	0.11	29.8	0.1	0.07	0.04	0.03	0.945
6542-11	05/14/02	0.11	29.6	0.1	0.05	0.1	0.02	0.96
6542-12	05/28/02	0.10	29.5	0.1	0.07	0.1	0.03	1.165

PANEL NUMBER	ORTHO VITROS ECi ANTI-HBc IgM	DIASORIN ANTI-HBe	ORTHO VITROS ECi ANTI-HBe	ABBOTT ANTI-HBs QUANT	ORTHO VITROS ECi ANTI-HBs QUANT	ORTHO VITROS ECi HBeAg	DIASORIN HBeAg
	TEST DATE: 8/21/2014	TEST DATE: UNK	TEST DATE: 8/21/2014	TEST DATE: UNK	TEST DATE: 8/21/2014	TEST DATE: 8/21/2014	TEST DATE: UNK
6542-01	3.61	0.1	1.76	0.4	1.5	0.80	0.31
6542-02	3.26	0.1	1.80	0.2	1.1	0.15	0.19
6542-03	2.35	0.2	1.48	0.3	1.8	0.47	0.28
6542-04	1.68	0.1	2.52	0.3	1.6	0.14	0.19
6542-05	1.16	0.07	5.03	0.4	1.4	0.13	0.07
6542-06	0.95	0.06	6.12	0.4	2.9	0.14	0.11
6542-07	0.65	0.09	3.29	0.6	5.9	0.15	0.05
6542-08	0.63	0.09	5.78	2.1	15.0	0.14	0.11
6542-09	0.48	0.1	2.15	2.4	25.9	0.14	0.09
6542-10	0.52	0.2	2.21	3.5	51.0	0.14	0.15
6542-11	0.54	0.2	2.45	4.8	84.2	0.13	0.01
6542-12	0.58	0.2	3.35	6.1	111	0.17	0.04

### Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)

PIHBV6542 Rev03  
Effective Date: 09/22/2021