



HBV Seroconversion Panel Donor No.64017
Product Code: HBV10232

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
 Donor was un-treated and asymptomatic throughout the collection period.

Panel Member	Bleed Date	VENDOR ALT	ZEPTOMETRIX HBV RT-PCR Copies/mL	VENDOR HBsAg	VENDOR Anti HIV 1/2	VENDOR HIV p24Ag	VENDOR Anti HCV	GENETICS SYSTEMS HBsAg
10232-01	04/26/97	9	178	Negative	Negative	Negative	Negative	Negative
10232-02	05/11/97	7	389	Negative	Negative	Negative	Negative	Negative
10232-03	05/13/97	9	389	Negative	Negative	Negative	Negative	Negative
10232-04	05/28/97	11	10,800	Positive	Negative	Negative	Negative	Positive
10232-05	05/31/97	5	20,500	Positive	Negative	Negative	Negative	Positive
10232-06	06/09/97	8	325,000	Positive	Negative	Negative	Negative	Positive

Panel Member	ABBOTT Auszyme Dynamic HBsAg	ORTHO VITROS ECi HBsAg	ORTHO VITROS ECi Anti HBc Total	ORTHO VITROS ECi HBeAg	ORTHO VITROS ECi Anti-HBe	ORTHO VITROS ECi Anti-HBc IgM	DIASORIN Anti-HBs	DIASORIN HBc IgM	ABBOTT Core-Total
10232-01	0.050	0.23	3.35	0.36	0.21	0.06	0.05	0.17	0.526
10232-02	0.233	0.35	3.26	0.21	0.16	0.05	0.03	0.17	0.463
10232-03	0.117	0.38	3.41	0.28	0.19	0.05	0.07	0.17	0.439
10232-04	2.667	14.2	3.12	0.22	0.16	0.05	0.06	0.17	0.543
10232-05	5.167	37.9	3.15	0.35	Not Tested	0.06	0.06	0.17	0.397
10232-06	>33.33	328	3.20	1.69	Not Tested	Not Tested	0.05	0.17	0.485

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.

This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.

ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
 For Customer Support, please visit www.zeptometrix.com or email custserv@zeptometrix.com