



**HBV Seroconversion Panel Donor No. 67303**  
**Part Number: HBV11008**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
 Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45u/L)	ABBOTT HBV REALTIME CP/ML	ROCHE CAP- G/CTM HBV (cp/mL)	GENETIC SYSTEMS HBsAg (2.0)	DADE BEHRING Enzygnost HBsAg 5.0	ORTHO HBsAg ELISA Testsystem 3	ABBOTT MUREX HBsAg Version 3	DIASORIN HBsAg
		TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 6/2/2005	TEST DATE: 7/14/2000	TEST DATE: 1/24/2000	TEST DATE: 4/18/2000	TEST DATE: UNK	TEST DATE: 10/24/1999
11008-01	03/09/98	11	NOT DETECTED	NEGATIVE	0.643	0.250	0.000	0.53	0.216
11008-02	03/16/98	10	NOT DETECTED	NEGATIVE	0.643	0.250	0.070	0.50	0.172
11008-03	04/06/98	6	NOT DETECTED	NEGATIVE	0.643	0.300	0.040	0.50	0.195
11008-04	04/09/98	7	NOT DETECTED	NEGATIVE	0.571	0.330	0.400	0.47	0.180
11008-05	04/14/98	10	NOT DETECTED	NEGATIVE	0.714	0.360	0.070	0.52	0.169
11008-06	04/16/98	14	NOT DETECTED	NEGATIVE	0.500	0.290	0.070	0.55	0.166
11008-07	04/21/98	10	NOT DETECTED	NEGATIVE	0.500	0.210	0.040	0.56	0.166
11008-08	04/26/98	3	NOT DETECTED	NEGATIVE	0.714	0.260	0.130	0.52	0.175
11008-09	04/29/98	6	<52	NEGATIVE	0.571	0.250	0.160	0.51	0.219
11008-10	05/03/98	7	<52	NEGATIVE	0.714	0.240	0.070	0.50	0.175
11008-11	05/06/98	7	58	NEGATIVE	0.571	0.240	0.070	0.50	0.166
11008-12	05/10/98	8	297	70.7	0.714	0.320	0.430	0.60	0.183
11008-13	05/13/98	4	290	844	0.857	0.470	0.190	0.78	0.231
11008-14	05/17/98	7	781	2020	0.786	0.730	0.700	1.50	0.243
11008-15	05/20/98	4	2055	6730	1.071	0.730	0.870	2.23	0.260
11008-16	05/27/98	6	8180	32600	5.286	8.680	9.880	14.10	0.769
11008-17	06/01/98	10	31824	83300	9.429	20.280	26.180	27.46	1.680
11008-18	06/03/98	7	91781	169000	14.214	31.350	41.410	29.33	2.083

PANEL MEMBER	ABBOTT Auszyme MC Dynamic HBsAg	ABBOTT Auszyme Monoclonal HBsAg	ABBOTT AxSYM HBsAg (V2) Reagent Pack	ABBOTT IMx HBsAg (V2) Reagent Pack	ORTHO VITROS HBsAg Reagent Pack	ROCHE ELECSYS HBsAg	ABBOTT PRISM HBsAg	SORIN ETI-MAK 3	SORIN ETI-MAK 4
	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 2/10/2000	TEST DATE: 2/24/2000	TEST DATE: 2/9/2000	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK
11008-01	0.100	0.073	0.570	0.500	0.080	0.470	0.37	0.080	0.200
11008-02	0.100	0.055	0.470	0.480	0.080	0.440	0.67	0.080	0.180
11008-03	0.100	0.073	0.480	0.3940	0.070	0.450	0.69	0.140	0.180
11008-04	0.100	0.073	0.430	0.390	0.070	0.530	0.66	0.120	0.280
11008-05	0.100	0.073	0.440	0.440	0.080	0.480	0.44	0.000	0.200
11008-06	0.140	0.073	0.550	0.420	0.070	0.560	0.50	0.250	0.180
11008-07	0.310	0.055	0.460	0.440	0.070	0.490	0.71	0.210	0.200
11008-08	0.100	0.055	0.470	0.420	0.060	0.510	0.62	0.190	0.230
11008-09	0.100	0.055	0.500	0.420	0.060	0.520	0.46	0.210	0.180
11008-10	0.100	0.091	0.500	0.430	0.070	0.580	0.59	0.190	0.230
11008-11	0.170	0.036	0.500	0.440	0.070	0.560	0.42	0.190	0.350
11008-12	0.140	0.091	0.520	0.460	0.080	0.580	0.79	0.190	0.300
11008-13	0.280	0.200	0.560	0.470	0.150	0.650	0.80	0.280	0.350
11008-14	0.830	0.455	0.670	0.590	0.310	0.920	1.64	0.410	0.800
11008-15	1.030	0.818	0.940	0.670	0.730	1.090	2.24	0.470	1.280
11008-16	11.520	7.455	4.520	2.960	12.800	6.690	17.14	3.740	12.230
11008-17	25.900	15.636	12.010	7.460	41.100	18.560	46.64	8.880	24.250
11008-18	35.380	33.273	16.930	10.640	69.00	30.810	63.85	15.480	37.180

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
 This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
 For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)

PIHBV11008 Rev03  
 Effective Date: 06/14/2021



**HBV Seroconversion Panel Donor No. 67303  
Part Number: HBV11008**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	ORTHO VITROS Eci HBsAg TEST DATE: 8/30/2018	ORTHO VITROS Eci ANTI-HCV TEST DATE: 8/30/2018	ORTHO VITROS Eci ANTI-HIV 1/2 TEST DATE: 8/30/2018	ABBOTT ANTI-HBc TOTAL TEST DATE: 9/1/1999	ORTHO VITROS Eci ANTI-HBc TOTAL TEST DATE: 8/30/2018	ABBOTT HBeAg TEST DATE: 9/1/1999	ORTHO VITROS Eci HBeAg TEST DATE: 8/30/2018
11008-01	0.05	0.02	0.11	NOT TESTED	2.35	NOT TESTED	0.12
11008-02	0.05	0.01	0.11	NOT TESTED	2.14	NOT TESTED	0.13
11008-03	0.04	0.01	0.12	NOT TESTED	2.48	NOT TESTED	0.13
11008-04	0.04	0.01	0.11	NOT TESTED	2.34	NOT TESTED	0.12
11008-05	0.04	0.01	0.11	NOT TESTED	2.31	NOT TESTED	0.13
11008-06	0.04	0.01	0.11	NOT TESTED	2.23	NOT TESTED	0.13
11008-07	0.04	0.01	0.12	NOT TESTED	2.28	NOT TESTED	0.12
11008-08	0.04	0.01	0.11	NOT TESTED	2.24	NOT TESTED	0.13
11008-09	0.29	0.01	0.12	NOT TESTED	2.29	NOT TESTED	0.24
11008-10	0.04	0.01	0.10	NOT TESTED	2.25	NOT TESTED	0.12
11008-11	0.04	0.01	0.10	NOT TESTED	2.53	NOT TESTED	0.12
11008-12	0.06	0.01	0.12	NOT TESTED	2.03	NOT TESTED	0.13
11008-13	0.08	0.01	0.12	NOT TESTED	2.42	NOT TESTED	0.13
11008-14	0.38	0.01	0.12	NOT TESTED	2.44	NOT TESTED	0.14
11008-15	0.84	0.01	0.12	NOT TESTED	2.58	NOT TESTED	0.13
11008-16	9.35	0.01	0.11	NOT TESTED	2.41	NOT TESTED	0.17
11008-17	27.5	0.01	0.11	NOT TESTED	2.26	NOT TESTED	0.33
11008-18	44.1	0.01	0.12	0.469	2.48	0.608	0.93

PANEL MEMBER	ORTHO VITROS Eci ANTI-HBe TEST DATE: 8/30/2018	ORTHO VITROS Eci ANTI-HBc IgM TEST DATE: 8/30/2018	ORTHO VITROS Eci ANTI-HBs TEST DATE: 8/30/2018
11008-01	0.18	0.03	1.3
11008-02	0.20	0.03	0.7
11008-03	0.19	0.03	0.4
11008-04	0.19	0.02	0.6
11008-05	0.19	0.03	1.6
11008-06	0.19	0.02	0.8
11008-07	0.19	0.03	0.5
11008-08	0.19	0.03	1.3
11008-09	0.18	0.03	0.7
11008-10	0.19	0.03	0.4
11008-11	0.19	0.02	0.3
11008-12	0.21	0.03	0.3
11008-13	0.19	0.02	0.6
11008-14	0.19	0.02	0.2
11008-15	0.20	0.02	0.1
11008-16	0.19	0.02	0.6
11008-17	0.19	0.03	0.2
11008-18	0.17	0.02	0.4

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)

PIHBV11008 Rev03  
Effective Date: 06/14/2021