

ZeptoMetrix®

HBV Seroconversion Panel Donor No. 67694 Part Number: HBV11011

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45u/L) TEST DATE: UNK	ABBOTT HBV REAL TIME CP/ML TEST DATE: UNK	ROCHE CAP-G/CTM HBV (cp/mL) TEST DATE: 6/2/2005	GENETIC SYSTEMS HBsAg (2.0) TEST DATE: UNK	DADE BEHRING Enzygnost HBsAg 5.0 TEST DATE: 1/24/2000	ORTHO HBsAg ELISA Testsystem 3 TEST DATE: UNK	ABBOTT MUREX HBsAg Version 3 TEST DATE: UNK
11011-01	03/14/98	9	NOT DETECTED	Negative	0.286	0.360	Negative	0.51
11011-02	04/30/98	9	NOT DETECTED	Negative	0.357	0.270	0.550	0.47
11011-03	05/02/98	7	NOT DETECTED	Negative	0.357	0.220	0.330	0.54
11011-04	05/07/98	3	NOT DETECTED	Negative	0.571	0.290	0.190	0.55
11011-05	05/09/98	1	NOT DETECTED	Negative	0.429	0.290	0.460	0.50
11011-06	05/16/98	5	NOT DETECTED	Negative	0.429	0.310	0.190	0.49
11011-07	05/21/98	6	NOT DETECTED	Negative	0.786	0.500	0.190	0.48
11011-08	05/23/98	7	NOT DETECTED	Negative	0.643	0.610	0.380	0.48
11011-09	06/25/98	8	1327	6610	0.929	0.810	0.930	1.04
11011-10	06/27/98	8	1559	5860	1.000	1.130	0.980	1.76
11011-11	07/02/98	4	9178	41100	2.786	4.200	3.900	4.90
11011-12	07/04/98	4	18313	86800	8.786	6.390	7.560	8.76
11011-13	07/09/98	6	66490	228000	15.000	24.830	30.360	27.72
11011-14	07/11/98	6	112916	366000	17.143	34.460	50.400	29.29

PANEL MEMBER	DIASORIN HBsAg TEST DATE: UNK	ABBOTT PRISM HBsAg TEST DATE: UNK	ROCHE ELECSYS HBsAg TEST DATE: 2/9/2000	ROCHE Cobas Core HBsAg II EIA TEST DATE: UNK	SORIN ETI-MAK 4 TEST DATE: UNK	ABBOTT HBeAg (OD) TEST DATE: 9/1/1999	ABBOTT ANTI-HBc TOTAL (OD) TEST DATE: 9/1/1999	ABBOTT Auszyme MC Dynamic HBsAg TEST DATE: 3/22/2000
11011-01	0.136	0.46	0.510	0.130	0.230	Not Tested	Not Tested	0.130
11011-02	0.104	0.42	0.550	0.120	0.200	Not Tested	Not Tested	0.090
11011-03	0.142	0.52	0.560	0.110	0.180	Not Tested	Not Tested	0.130
11011-04	0.157	0.60	0.520	0.110	0.180	Not Tested	Not Tested	0.160
11011-05	0.157	0.49	0.480	0.120	0.150	Not Tested	Not Tested	0.220
11011-06	0.151	0.56	0.560	0.130	0.180	Not Tested	Not Tested	0.000
11011-07	0.127	0.42	0.490	0.120	0.250	Not Tested	Not Tested	0.060
11011-08	0.154	0.36	0.540	0.140	0.280	Not Tested	Not Tested	0.060
11011-09	0.219	1.80	0.820	0.820	0.750	Not Tested	Not Tested	0.470
11011-10	0.166	3.35	1.050	1.080	1.230	Not Tested	Not Tested	0.510
11011-11	0.269	10.93	2.650	4.200	11.230	Not Tested	Not Tested	4.230
11011-12	0.550	18.51	4.430	8.500	8.250	Not Tested	Not Tested	8.340
11011-13	1.876	65.36	17.400	37.100	32.580	Not Tested	Not Tested	26.120
11011-14	2.633	94.81	27.310	56.900	47.080	0.08	0.84	54.000

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for Anti-HIV1/2, HIV p24 Ag, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHBV11011 Rev03
Effective Date: 08/12/2021



**HBV Seroconversion Panel Donor No. 67694
Part Number: HBV11011**

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	ABBOTT Auszyme Monoclonal HBsAg TEST DATE: UNK	ORTHO VITROS HBsAg Reagent Pack TEST DATE: 2/24/2000	ABBOTT AxSYM HBsAg (V2) Reagent Pack TEST DATE: 2/28/2000	ORTHO VITROS Eci HBsAg TEST DATE: 8/30/2018	ORTHO VITROS Eci ANTI-HBc TOTAL TEST DATE: 8/30/2018	ORTHO VITROS Eci HBeAg TEST DATE: 8/30/2018	ORTHO VITROS Eci ANTI-HBe TEST DATE: 8/30/2018	ORTHO VITROS Eci ANTI-HBc IgM TEST DATE: 8/30/2018	ORTHO VITROS Eci ANTI-HBs QUANT TEST DATE: 8/30/2018
11011-01	0.019	0.140	0.320	0.07	3.08	0.14	0.16	0.02	0.0
11011-02	0.038	0.130	0.380	0.08	3.25	0.14	0.17	0.02	0.0
11011-03	0.038	0.130	0.370	0.08	3.34	0.14	0.17	0.02	0.0
11011-04	0.057	0.120	0.370	0.26	3.46	0.14	0.16	0.02	0.0
11011-05	0.057	0.130	0.390	0.07	3.54	0.14	0.17	0.01	0.0
11011-06	0.019	0.130	0.380	0.06	3.64	0.13	0.16	0.01	0.0
11011-07	0.094	0.140	0.370	0.06	3.32	0.15	0.16	0.02	0.0
11011-08	0.019	0.130	0.500	0.06	3.34	0.14	0.16	0.01	0.0
11011-09	0.604	0.900	0.630	0.65	3.35	0.14	0.17	0.02	0.0
11011-10	0.774	1.240	0.670	1.59	3.45	0.15	0.21	0.02	0.0
11011-11	3.660	8.140	1.740	6.73	3.34	0.18	0.17	0.02	0.0
11011-12	6.566	15.700	3.170	13.5	3.39	0.22	0.17	0.02	0.0
11011-13	26.038	71.400	9.500	56.1	3.38	1.24	0.16	0.02	0.0
11011-14	36.792	117.000	15.880	89.3	3.53	2.64	0.14	0.01	0.0

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for Anti-HIV1/2, HIV p24 Ag, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com