

ZeptoMetrix®

HBV Seroconversion Panel Donor No. 67773 Part Number: HBV11012

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45u/L) TEST DATE: 7/31/1998	ABBOTT HBV REAL TIME CP/ML TEST DATE: UNK	ROCHE CAP-G/CTM HBV (cp/mL) TEST DATE: 6/2/2005	GENETIC SYSTEMS HBsAg (2.0) TEST DATE: UNK	DADE BEHRING Enzygnost HBsAg 5.0 TEST DATE: 1/24/2000	ROCHE Cobas Core HBsAg II EIA TEST DATE: UNK	DIASORIN HBsAg TEST DATE: UNK
11012-01	06/27/98	3	<52	NEGATIVE	0.385	0.390	0.160	0.112
11012-02	06/29/98	4	<52	NEGATIVE	0.321	0.300	0.170	0.160
11012-03	07/05/98	12	149	NEGATIVE	0.256	0.400	0.190	0.183
11012-04	07/15/98	4	2305	5200	0.769	1.040	0.490	0.183
11012-05	07/17/98	8	2709	10400	0.833	1.530	0.930	0.225
11012-06	07/22/98	5	5404	24500	2.051	4.710	3.210	0.589

PANEL MEMBER	ROCHE ELECSYS HBsAg TEST DATE: 2/9/2000	ABBOTT PRISM HBsAg TEST DATE: UNK	ABBOTT MUREX HBsAg Version 3 TEST DATE: UNK	ORTHO HBsAg ELISA Testsystem 3 TEST DATE: UNK	SORIN ETI-MAK 4 TEST DATE: UNK	ABBOTT HBeAg (OD) TEST DATE: 9/8/1999	ABBOTT ANTI-HBc TOTAL (OD) TEST DATE: 9/8/1999	ABBOTT Auszyme MC Dynamic HBsAg TEST DATE: 3/22/2000
11012-01	0.450	0.68	0.48	0.190	0.180	NOT TESTED	NOT TESTED	0.190
11012-02	0.440	0.56	0.50	0.270	0.330	NOT TESTED	NOT TESTED	0.160
11012-03	0.530	0.72	0.56	0.270	0.200	NOT TESTED	NOT TESTED	0.250
11012-04	0.930	2.23	1.19	0.790	1.030	NOT TESTED	NOT TESTED	0.850
11012-05	1.490	3.69	1.98	1.670	2.330	NOT TESTED	NOT TESTED	1.770
11012-06	4.420	11.86	5.79	3.850	7.230	0.04	0.09	4.990

PANEL MEMBER	ABBOTT Auszyme Monoclonal HBsAg TEST DATE: UNK	ABBOTT AxSYM HBsAg (V2) Reagent Pack TEST DATE: 2/28/2000	ORTHO VITROS ECi HBsAg TEST DATE: 8/30/2018	ORTHO VITROS ECi ANTI-HBc TOTAL TEST DATE: 8/30/2018	ORTHO VITROS ECi HBeAg TEST DATE: 8/30/2018	ORTHO VITROS ECi ANTI-HBe TEST DATE: 8/30/2018	ORTHO VITROS ECi ANTI-HBc IgM TEST DATE: 8/30/2018	ORTHO VITROS ECi ANTI-HBs QUANT TEST DATE: 8/30/2018
11012-01	0.038	0.390	0.06	3.18	0.13	0.20	0.03	73.4
11012-02	0.019	0.380	0.05	3.21	0.13	0.18	0.02	54.2
11012-03	0.096	0.370	0.07	2.85	0.14	0.18	0.02	53.7
11012-04	0.577	0.750	0.53	2.89	0.13	0.21	0.03	64.8
11012-05	1.192	0.980	1.14	3.03	0.20	0.17	0.02	43.2
11012-06	3.154	2.380	5.56	3.05	0.24	0.17	0.02	45.8

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for Anti-HIV1/2, HIV p24 Ag, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHBV11012 Rev03
Effective Date: 08/12/2021