



**HBV Seroconversion Panel Donor No. 68541
Part Number: HBV11016**

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45u/L) TEST DATE: UNK	ABBOTT HBV REAL TIME CP/ML TEST DATE: UNK	ABBOTT ANTI-HBc TOTAL (OD) TEST DATE: 9/8/1999	GENETIC SYSTEMS HBsAg (2.0) TEST DATE: UNK	DADE BEHRING Enzygnost HBsAg 5.0 TEST DATE: 12/15/1999	ABBOTT PRISM HBsAg TEST DATE: UNK	BIOTEST ELISA HBsAg TEST DATE: 4/18/2000
11016-01	08/23/98	6	NOT DETECTED	Not Tested	0.625	0.950	0.36	0.160
11016-02	08/25/98	3	NOT DETECTED	Not Tested	0.563	0.230	0.39	0.180
11016-03	08/31/98	6	NOT DETECTED	Not Tested	0.813	0.230	0.33	0.220
11016-04	09/07/98	5	103	Not Tested	0.563	0.220	0.36	0.240
11016-05	09/10/98	7	156	Not Tested	0.375	0.280	0.65	0.160
11016-06	09/19/98	8	2709	Not Tested	0.688	1.240	2.93	0.390
11016-07	09/22/98	7	6804	Not Tested	2.000	3.060	5.97	0.540
11016-08	09/28/98	8	Not Tested	Not Tested	2.813	16.110	5.04	4.050
11016-09	10/01/98	2	220168	Not Tested	9.813	37.190	71.83	10.950
11016-10	10/06/98	12	856553	0.97	14.375	OVER	175.62	40.220

PANEL MEMBER	ABBOTT Auszyme MC Dynamic HBsAg TEST DATE: 12/15/1999	ABBOTT Auszyme Monoclonal HBsAg TEST DATE: UNK	ORTHO HBsAg ELISA Testsystem 3 TEST DATE: 1/4/2000	ABBOTT MUREX HBsAg Version 3 TEST DATE: UNK	ROCHE Cobas Core HBsAg II EIA TEST DATE: 4/18/2000	BIORAD MONALISA HBsAg PLUS TEST DATE: UNK	ABBOTT AxSYM HBsAg (V2) Reagent Pack TEST DATE: 1/10/2000	ABBOTT HBeAg TEST DATE: 9/8/1999
11016-01	0.140	0.038	0.310	0.52	0.180	0.280	0.590	Not Tested
11016-02	0.200	0.019	0.050	0.52	0.160	0.480	0.500	Not Tested
11016-03	0.340	0.038	0.150	0.51	0.160	0.320	0.530	Not Tested
11016-04	0.860	0.038	0.120	0.63	0.180	0.300	0.500	Not Tested
11016-05	0.110	0.077	0.120	0.64	0.180	0.420	0.560	Not Tested
11016-06	1.380	0.846	1.180	1.93	1.050	1.000	1.010	Not Tested
11016-07	2.010	2.500	2.370	3.89	2.400	1.930	1.970	Not Tested
11016-08	18.110	11.731	15.060	17.68	13.830	8.520	8.510	Not Tested
11016-09	51.880	37.942	41.470	28.41	38.960	18.200	20.400	Not Tested
11016-10	57.420	>38.462	OVER	28.86	70.470	52.970	57.530	0.818

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for Anti-HIV1/2, HIV p24 Ag, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHBV11016 Rev03
Effective Date: 08/12/2021



**HBV Seroconversion Panel Donor No. 68541
Part Number: HBV11016**

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	SORIN ETI-MAK 3 TEST DATE: 12/8/1999	SORIN ETI-MAK 4 TEST DATE: UNK	ORTHO VITROS ECi HBsAg TEST DATE: 8/30/2018	ORTHO VITROS ECi ANTI-HBc TOTAL TEST DATE: 8/30/2018	ORTHO VITROS ECi HBeAg TEST DATE: 8/30/2018	ORTHO VITROS ECi ANTI-HBe TEST DATE: 8/30/2018	ORTHO VITROS ECi ANTI-HBc IgM TEST DATE: 8/30/2018	ORTHO VITROS ECi ANTI-HBs QUANT TEST DATE: 8/30/2018
11016-01	0.020	0.130	0.18	2.82	0.13	0.17	0.01	0.0
11016-02	0.000	0.180	0.06	3.08	0.13	0.17	0.02	0.0
11016-03	0.070	0.150	0.07	3.08	0.13	0.17	0.01	0.0
11016-04	0.070	0.200	0.11	3.16	0.13	0.17	0.02	0.0
11016-05	0.090	0.230	0.13	2.93	0.13	0.19	0.01	0.0
11016-06	0.780	1.380	1.08	2.86	0.13	Not Tested	0.02	0.0
11016-07	1.860	3.600	2.61	2.99	0.14	Not Tested	0.01	0.0
11016-08	10.180	21.100	Not Tested	Not Tested	Not Tested	Not Tested	Not Tested	Not Tested
11016-09	28.710	44.980	55.0	2.83	0.90	Not Tested	0.01	0.0
11016-10	OVER	82.930	263	3.07	6.59	0.13	0.01	0.0

www.h-h-c.com

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for Anti-HIV1/2, HIV p24 Ag, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com