

# ZeptoMetrix®

## HBV Seroconversion Panel Donor No. 71415 Part Number: HBV11027

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45u/L)  TEST DATE: 9/23/1999	ABBOTT HBV REAL TIME CP/ML  TEST DATE: UNK	ROCHE HBV Monitor Assay (copies/mL)  TEST DATE: UNK	ROCHE HBV Ampliscreen  TEST DATE: UNK	GENETIC SYSTEMS HBsAg 2.0 Static  TEST DATE: UNK	VENDOR HBsAg  TEST DATE: UNK	ABBOTT PRISM HBsAg  TEST DATE: UNK	ABBOTT MUREX HBsAg Version 3  TEST DATE: UNK
11027-01	03/16/99	3	NOT DETECTED	NOT DETECTED	NEGATIVE	NEGATIVE	NEGATIVE	0.45	0.52
11027-02	03/21/99	14	NOT DETECTED	NOT DETECTED	NEGATIVE	NEGATIVE	NEGATIVE	0.48	0.55
11027-03	03/28/99	11	NOT DETECTED	<100	POSITIVE	NEGATIVE	NEGATIVE	0.48	0.51
11027-04	03/30/99	12	NOT DETECTED	NOT DETECTED	NEGATIVE	NEGATIVE	NEGATIVE	0.43	0.50
11027-05	04/04/99	7	<52	100	POSITIVE	NEGATIVE	NEGATIVE	0.64	0.51
11027-06	04/06/99	10	<52	200	POSITIVE	NEGATIVE	NEGATIVE	0.65	0.53
11027-07	04/11/99	9	59	200	POSITIVE	NEGATIVE	NEGATIVE	0.60	0.63
11027-08	04/14/99	8	277	200	POSITIVE	NEGATIVE	NEGATIVE	0.73	0.71
11027-09	04/18/99	12	540	500	POSITIVE	NEGATIVE	NEGATIVE	1.24	0.91
11027-10	04/22/99	12	1749	<100	POSITIVE	NEGATIVE	NEGATIVE	2.64	1.69
11027-11	04/25/99	8	3410	12000	POSITIVE	NEGATIVE	NEGATIVE	6.11	3.55
11027-12	05/15/99	11	NOT TESTED	4600000	POSITIVE	POSITIVE	POSITIVE	361.69	33.44
11027-13	05/19/99	12	17488573	45000000	POSITIVE	POSITIVE	POSITIVE	415.88	33.05
11027-14	05/23/99	4	63497171	80000000	POSITIVE	POSITIVE	POSITIVE	551.58	33.35

PANEL MEMBER	ORTHO HBsAg Proc B  TEST DATE: UNK	ABBOTT HBsAg Proc B EIA  TEST DATE: UNK	ABBOTT HBsAg Proc C  TEST DATE: UNK	ORTHO VITROS ECi HBsAg  TEST DATE: 8/30/2018	ORTHO VITROS ECi ANTI-HBc TOTAL  TEST DATE: 8/30/2018	ORTHO VITROS ECi HBeAg  TEST DATE: 8/30/2018	ORTHO VITROS ECi ANTI-HBe  TEST DATE: 8/30/2018	ORTHO VITROS ECi ANTI-HBc IgM  TEST DATE: 8/30/2018	ORTHO VITROS ECi ANTI-HBs QUANT  TEST DATE: 8/30/2018
11027-01	0.033	0.038	0.172	0.06	3.09	0.12	0.17	0.03	0.1
11027-02	0.033	0.038	0.069	0.06	3.18	0.14	0.17	0.02	0.4
11027-03	0.000	0.057	0.069	0.10	2.93	0.14	0.18	0.03	1.1
11027-04	0.000	0.094	0.034	0.05	3.11	0.14	0.18	0.02	0.3
11027-05	0.000	0.019	0.103	0.05	2.96	0.12	0.17	0.03	0.0
11027-06	0.033	0.019	0.241	0.19	3.32	0.14	0.17	0.02	1.0
11027-07	0.033	0.038	0.172	0.09	3.39	0.14	0.18	0.03	0.0
11027-08	0.033	0.038	0.138	0.07	3.39	0.13	0.20	0.03	0.9
11027-09	0.100	0.245	0.483	0.19	3.31	0.14	0.17	0.03	0.0
11027-10	0.267	0.868	0.241	0.85	3.18	0.16	0.19	0.03	0.0
11027-11	0.667	4.528	1.138	2.49	3.04	0.14	0.16	0.02	0.5
11027-12	104.333	>37.736	75.862	NOT TESTED	NOT TESTED	NOT TESTED	NOT TESTED	NOT TESTED	NOT TESTED
11027-13	93.433	>37.736	75.862	3780	2.78	769	0.01	0.04	0.2
11027-14	82.767	>37.736	75.862	6310	2.29	2120	0.00	0.04	3.7

### Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for Anti-HIV1/2, HIV p24 Ag, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)

PIHBV11027 Rev04  
Effective Date: 06/15/2021