

ZeptoMetrix®

HBV Seroconversion Panel Donor No.61042 Part Number: HBV6273

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45u/L) TEST DATE: 3/01/1996	ABBOTT HBV REAL TIME CP/ML TEST DATE: UNK	BIOMERIEUX VIDAS HBsAg TEST DATE: 1/16/1997	BOEHR. MANNHEIM ELECSYS HBsAg TEST DATE: 10/9/1997	ORGANON TEKNIKA Hepanostika HBsAg Uni-Form II TEST DATE: 12/18/1996	ORTHO VITROS ECI HBsAg ELISA Testsystem 3 TEST DATE: 12/16/1996
6273-01	01/29/96	12	NOT DETECTED	0.21	0.40	0.50	0.26
6273-02	02/01/96	8	<52	0.12	0.39	0.59	0.06
6273-03	02/05/96	12	136	0.16	0.39	0.48	0.23
6273-04	02/12/96	15	1422	0.42	0.64	0.54	0.38
6273-05	02/23/96	16	20080	6.75	7.33	1.91	7.28
6273-06	02/28/96	10	215156	39.17	61.29	10.48	55.07

PANEL MEMBER	MUREX HBsAg TEST DATE: 1/9/1997	SORIN ETI- MAK 3 HBsAg TEST DATE: 1/16/1997	ABBOTT PRISM HBsAg TEST DATE: 8/27/2009	ABBOTT Murex HBsAg Version 3 TEST DATE: 8/27/2009	ORTHO VITROS ECI AMERLITE HBsAg II TEST DATE: 1/13/1997	ABBOTT AxSYM HBsAg Reagent Pack TEST DATE: 1/16/1997	ABBOTT Austria II-125 TEST DATE: 12/3/1996	ABBOTT Anti-HBc TOTAL (EIA) TEST DATE: 12/17/1996
6273-01	0.52	0.15	0.57	0.48	0.07	0.67	0.44	0.490
6273-02	0.51	0.18	1.15	0.55	0.07	0.49	0.32	0.463
6273-03	0.65	0.18	1.62	0.53	0.08	0.53	0.47	0.447
6273-04	0.62	0.42	1.66	0.92	0.17	1.10	0.57	0.486
6273-05	2.36	7.08	23.06	7.67	5.81	5.19	3.46	0.426
6273-06	12.11	34.85	118.26	34.13	42.48	26.47	17.55	0.534

PANEL MEMBER	ABBOTT Auszyme MC Dynamic TEST DATE: 1/21/1997	DIASORIN HBc IgM TEST DATE: 10/5/2011	ABBOTT Auszyme Monoclonal TEST DATE: 9/30/1997	ABBOTT HBsAg (EIA) TEST DATE: 11/19/1996	ABBOTT IMx HBsAg Reagent Pack TEST DATE: 3/13/1997	BIOTEST ELISA HBsAg TEST DATE: 3/3/1997	SANOFI Monalisa HBsAg 2 nd Generation TEST DATE: 3/6/1997	ROCHE CAP-G/CTM HBV (cp/mL) TEST DATE: 6/2/2005
6273-01	0.21	0.17	0.15	0.067	0.52	0.32	0.38	NEGATIVE
6273-02	0.30	0.17	0.13	0.117	0.51	0.20	0.47	NEGATIVE
6273-03	0.27	0.21	0.15	0.117	0.47	0.22	0.51	NOT TESTED
6273-04	0.76	0.17	0.34	0.300	0.93	0.34	0.58	5090
6273-05	13.00	0.17	6.47	5.800	3.21	4.57	3.87	69200
6273-06	OVER	0.17	36.73	>33.333	12.49	26.95	24.8	416000

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for Anti-HIV1/2, HIV p24 Ag, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHBV6273 Rev03
Effective Date: 06/23/2021



**HBV Seroconversion Panel Donor No.61042
Part Number: HBV6273**

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	BEHRINGWERKE Enzygnost HBsAg Monoclonal II TEST DATE: 12/19/1996	ORTHO VITROS ECi HBsAg TEST DATE: 8/30/2018	ORTHO VITROS ECi ANTI-HBc TOTAL TEST DATE: 8/30/2018	ORTHO VITROS ECi HBeAg TEST DATE: 8/30/2018	ORTHO VITROS ECi ANTI-HBe TEST DATE: 8/30/2018	ORTHO VITROS ECi ANTI-HBc IgM TEST DATE: 8/30/2018	ORTHO VITROS ECi ANTI-HBs QUANT TEST DATE: 8/30/2018
6273-01	0.74	0.06	2.78	0.19	0.20	0.02	0.0
6273-02	0.45	0.12	2.64	0.20	0.21	0.02	0.0
6273-03	0.36	0.10	2.69	0.19	0.20	0.02	0.0
6273-04	0.87	0.22	3.05	0.19	0.20	0.02	0.0
6273-05	3.13	9.97	2.90	0.30	0.19	0.03	0.0
6273-06	16.88	76.1	2.85	1.49	0.19	0.03	0.0

www.h-h-c.com

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for Anti-HIV1/2, HIV p24 Ag, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com