



HBV Seroconversion Panel Donor No.63426
Part Number: HBV6278

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
 Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45u/L)	ABBOTT HBV REAL TIME CP/ML	RESEARCH USE PCR	ROCHE CAP-G/CTM HBV (cp/mL)	BIOMERIEUX VIDAS HBsAg (>0.13 reactive)	BOEHR. MANNHEIM HBsAg Enzymum-test ES Systems	ORGANON TEKNIKA Hepanostika HBsAg Uni-Form II	ORTHO HBsAg ELISA Testsystem 3
		TEST DATE: 3/25/1997	TEST DATE: UNK	TEST DATE: 6/14/1998	TEST DATE: 6/2/2005	TEST DATE: 10/21/1997	TEST DATE: 10/9/1997	TEST DATE: 10/7/1997	TEST DATE: 10/30/1997
6278-01	02/10/97	9	65	100	400	0.02	0.34	0.44	0.38
6248-02	02/14/97	3	410	100	1510	0.03	0.55	0.49	0.36
6278-03	02/18/97	8	3332	500	15300	0.05	0.59	0.47	0.41
6278-04	02/22/97	8	12381	1000	50300	0.26	1.69	0.86	0.44
6278-05	02/26/97	6	93919	40000	349000	1.40	6.42	2.35	0.38
6278-06	03/01/97	8	504376	120000	1300000	6.32	32.64	10.32	0.33
6278-07	03/05/97	1	3570708	1200000	17000000	12.55	OVER	25.04	0.52
6278-08	03/08/97	0	18312784	6900000	32000000	13.62	OVER	27.24	1.48
6278-09	03/15/97	15	896921385	50000000	>110000000	12.76	OVER	26.69	22.24
6278-10	03/19/97	4	>3410000000	>50000000	>110000000	11.25	OVER	27.19	12.08
6278-11	03/23/97	10	>3410000000	>50000000	>110000000	14.06	OVER	27.20	22.57

PANEL MEMBER	SORIN ETI-MAK 3 HBsAg	MUREX HBsAg	ABBOTT PRISM HBsAg	ABBOTT MUREX HBsAg Version 3	BIOTEST ELISA HBsAg	ABBOTT AxSYM HBsAg	ABBOTT HBsAg (EIA)	ORTHO VITROS ECi HBsAg	BOEHR. MANNHEIM ELECSYS HBsAg
	TEST DATE: 11/12/1997	TEST DATE: 10/29/1997	TEST DATE: 8/27/2009	TEST DATE: 8/27/2009	TEST DATE: 11/5/1997	TEST DATE: 10/22/1997	TEST DATE: 12/5/1997	TEST DATE: UNK	TEST DATE: 11/8/1997
6278-01	0.23	0.48	0.61	0.51	0.14	0.40	0.071	0.27	0.38
6278-02	0.32	0.47	1.61	0.61	0.22	0.53	0.095	0.26	0.48
6278-03	0.75	0.53	1.27	0.86	0.21	0.60	0.100	0.23	0.65
6278-04	1.90	1.89	5.54	2.69	1.02	1.49	0.095	0.24	1.58
6278-05	7.74	9.04	30.76	14.82	5.15	5.58	0.119	0.24	8.46
6278-06	31.17	17.09	101.55	32.55	17.10	22.57	0.143	0.25	34.37
6278-07	37.16	OVER	199.08	33.49	24.39	53.27	0.167	0.34	182.0
6278-08	39.72	OVER	315.50	33.21	24.94	74.90	0.214	0.91	913.3
6278-09	40.20	OVER	497.70	33.13	23.38	117.10	1.00	14.6	1503.0
6278-10	38.20	OVER	743.42	33.38	23.85	126.75	2.34	24.4	3723.0
6278-11	39.63	OVER	675.63	33.26	23.79	99.01	4.8	25.8	629.10

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for Anti-HIV1/2, HIV p24 Ag, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
 This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
 For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHBV6278 Rev05
 Effective Date: 06/23/2021



**HBV Seroconversion Panel Donor No.63426
Part Number: HBV6278**

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	ABBOTT IMx HBsAg Reagent Pack TEST DATE: 10/30/1997	BEHRINGW ERKE Enzygnost HBsAg Monoclonal II TEST DATE: 10/16/1997	ORTHO VITROS ECI ANTI-HBs QUANT TEST DATE: UNK	ABBOTT ANTI-HBs QUANT TEST DATE: 12/5/1997	ORTHO VITROS ECI ANTI-HBc IgM TEST DATE: UNK	ORTHO VITROS ECI ANTI-HBc TOTAL TEST DATE: UNK	ABBOTT ANTI-HBc TOTAL TEST DATE: 12/5/1997	ABBOTT ANTI-HBc IgM TEST DATE: 6/18/1998
6278-01	0.48	0.36	2.6	0.111	0.06	2.72	0.075	0.082
6278-02	0.52	0.37	1.1	0.143	0.05	2.85	0.078	0.077
6278-03	0.45	0.43	1.4	0.159	0.05	3.09	0.057	0.079
6278-04	1.13	0.79	2.9	0.127	0.04	2.99	0.057	0.069
6278-05	4.06	3.71	1.4	0.175	0.05	2.88	0.065	0.059
6278-06	16.23	17.51	3.5	0.206	0.05	3.03	0.092	0.074
6278-07	37.47	37.30	1.7	0.127	0.05	2.81	0.050	0.085
6278-08	56.43	OVER	0.8	0.079	0.04	2.86	0.023	0.077
6278-09	82.25	OVER	1.4	0.175	0.22	1.06	0.331	0.336
6278-10	88.30	OVER	Not Tested	0.159	Not Tested	0.24	0.799	0.928
6278-11	72.76	38.28	2.6	0.206	2.79	0.05	1.782	3.615

PANEL MEMBER	ORTHO VITROS ECI HBeAg TEST DATE: UNK	ABBOTT HBeAg TEST DATE: 6/19/1998	ORTHO VITROS ECI ANTI-HBe TEST DATE: UNK
6278-01	0.23	0.270	0.17
6278-02	0.28	0.360	0.15
6278-03	0.26	0.258	0.14
6278-04	0.36	0.348	0.14
6278-05	2.07	0.573	0.13
6278-06	6.82	1.393	0.12
6278-07	105	5.607	0.04
6278-08	517	8.921	0.01
6278-09	2920	10.022	0.00
6278-10	Not Tested	13.966	Not Tested
6278-11	2430	15.123	0.00

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for Anti-HIV1/2, HIV p24 Ag, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHBV6278 Rev05
Effective Date: 06/23/2021