



HBV Seroconversion Panel Donor No. 63253
Part Number: HBV6287

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
 Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45u/L)	ABBOTT HBV REAL TIME CP/ML	RESEARCH USE PCR (copies/ml)	BIOMERIEUX VIDAS HBsAg	ORGANON TEKNIKA Hepanostika HBsAg Uni-Form II	ORTHO HBsAg ELISA Testsystem 3	ORTHO VITROS ECI HBsAg
		TEST DATE: 2/2/1998	TEST DATE: UNK	TEST DATE: 6/17/1998	TEST DATE: 4/30/1998	TEST DATE: 4/23/1998	TEST DATE: 3/18/1998	TEST DATE: 8/30/2018
6287-01	12/10/96	4	NOT DETECTED	<100	0.13	0.41	0.10	0.06
6287-02	12/12/96	7	NOT DETECTED	<100	0.13	0.42	0.01	0.07
6287-03	01/09/97	3	NOT DETECTED	<100	0.11	0.58	0.01	0.07
6287-04	01/13/97	4	NOT DETECTED	<100	0.13	0.72	0.04	0.08
6287-05	01/16/97	9	NOT DETECTED	<100	0.17	0.71	0.04	0.17
6287-06	01/20/97	6	<52	<100	0.14	0.62	0.04	0.20
6287-07	01/23/97	6	<52	<100	0.13	0.36	0.01	0.06
6287-08	01/28/97	10	374	100	0.18	0.56	0.07	0.07
6287-09	01/30/97	6	680	500	0.19	0.46	0.15	0.09
6287-10	02/16/97	5	200796	46000	23.24	12.27	41.66	44.9
6287-11	02/24/97	5	4816753	600000	130.53	MAX.ABS.	MAX.ABS.	907

PANEL MEMBER	GENETIC SYSTEMS HBsAg (2.0)	BIOTEST ELISA HBsAg	ABBOTT AxSYM Core-M	ABBOTT AxSYM Core (Co/S)	ABBOTT ANTI-HBc TOTAL	ORTHO VITROS ECI ANTI-HBc TOTAL	ABBOTT ANTI-HBc IgM	ORTHO VITROS ECI ANTI-HBc IgM
	TEST DATE: 2/6/1998	TEST DATE: 4/6/1998	TEST DATE: 4/7/1998	TEST DATE: 4/7/1998	TEST DATE: UNK	TEST DATE: 8/30/2018	TEST DATE: UNK	TEST DATE: 8/30/2018
6287-01	0.187	0.32	0.06	0.47	0.49	3.67	0.182	0.02
6287-02	0.168	0.27	0.06	0.49	0.45	3.92	0.182	0.02
6287-03	0.168	0.37	0.08	0.48	0.48	3.35	0.200	0.02
6287-04	0.243	0.28	0.07	0.49	0.39	3.56	0.182	0.02
6287-05	0.159	0.33	0.07	0.49	0.56	3.44	0.182	0.02
6287-06	0.187	0.25	0.07	0.48	0.48	3.84	0.164	0.02
6287-07	0.206	0.26	0.05	0.48	0.37	3.60	0.164	0.01
6287-08	0.196	0.21	0.06	0.50	0.45	3.68	0.182	0.02
6287-09	0.271	0.31	0.05	0.51	0.51	3.43	0.182	0.02
6287-10	8.860	10.09	0.07	0.51	0.48	3.52	0.182	0.02
6287-11	25.757	30.90	0.07	0.49	0.48	3.55	0.182	0.02

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for Anti-HIV1/2, HIV p24 Ag, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
 This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
 For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com



HBV Seroconversion Panel Donor No. 63253
Part Number: HBV6287

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
 Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	ABBOTT Auszyme Monoclonal HBsAg TEST DATE: 5/5/1998	ABBOTT PRISM HBsAg TEST DATE: 8/27/2009	ABBOTT MUREX HBsAg Version 3 TEST DATE: 8/27/2009	ABBOTT AxSYM HBsAg Reagent Pack TEST DATE: 3/19/1998	ABBOTT Auszyme MC Dynamic HBsAg TEST DATE: 4/6/1998	ORTHO VITROS ECi HBeAg TEST DATE: 8/30/2018	ORTHO VITROS ECi ANTI-HBe TEST DATE: 8/30/2018
6287-01	0.15	0.44	0.50	0.51	0.07	0.19	0.21
6287-02	0.21	0.45	0.54	0.54	0.03	0.19	0.22
6287-03	0.34	0.48	0.50	0.52	0.07	0.17	0.21
6287-04	0.52	0.43	0.49	0.44	0.29	0.20	0.21
6287-05	0.49	0.44	0.78	0.46	0.01	0.20	0.22
6287-06	0.31	0.40	0.62	0.49	0.23	0.19	0.20
6287-07	0.24	0.40	0.55	0.52	0.42	0.20	0.18
6287-08	0.24	0.53	0.54	0.52	0.20	0.20	0.19
6287-09	0.24	0.60	0.58	0.48	0.03	0.19	0.22
6287-10	55.99	48.24	25.66	10.01	42.95	2.22	0.16
6287-11	MAX.ABS.	228.08	30.77	51.44	MAX.ABS.	122	0.05

PANEL MEMBER	ORTHO VITROS ECi ANTI-HBs QUANT TEST DATE: 8/30/2018	SORIN (INCSTAR) HBsAg TEST DATE: UNK	BEHRINGWERKE Enzygnost HBsAg Monoclonal II TEST DATE: 5/4/1998	BOEHR. MANNHEIM HBsAg Ensymun-Test ES Systems TEST DATE: 4/27/1998	ROCHE Cobas Core HBsAg II EIA TEST DATE: 4/23/1998	SORIN ETI-Core IGMK2 TEST DATE: 5/11/1998	SORIN ETI-AB CoreK2 (Co/S) TEST DATE: 5/7/1998
6287-01	0.6	0.116	0.28	0.00	0.28	0.30	0.18
6287-02	2.3	0.116	0.28	0.05	0.20	0.29	0.17
6287-03	1.3	0.173	0.32	0.13	0.22	0.34	0.16
6287-04	0.2	0.116	0.27	0.00	0.05	0.32	0.18
6287-05	0.8	0.173	0.24	0.05	0.48	0.27	0.19
6287-06	0.5	0.289	0.24	0.00	0.20	0.25	0.19
6287-07	0.0	0.116	0.08	0.17	0.24	0.30	0.18
6287-08	0.0	0.173	0.29	0.17	0.22	0.34	0.19
6287-09	1.0	0.173	0.37	0.10	0.50	0.28	0.18
6287-10	0.0	6.012	12.88	13.70	30.04	0.31	0.17
6287-11	0.0	9.191	37.65	234.5	64.81	0.26	0.17

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for Anti-HIV1/2, HIV p24 Ag, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
 This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
 For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com