



HBV Seroconversion Panel Donor No.74302
Part Number: HBV9098

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
 Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	(QUEST) ROCHE	FUJIREBIO	ROCHE	ABBOTT	DIASORIN	SIEMENS
		REAL TIME PCR IU/mL	LUMIPULSE G HBsAg-QUANT (233122) G1200 miU/MI	HBsAg II (04687787190) COBAS e411 COi	ARCHITECT HBsAg QUALITATIVE II (2G22) i200 S/CO	MUREX HBsAg Version 3 (GE36) S/CO	ENZYGNOST HBsAg 6.0 (OPFM055) BEP III S/CO
		TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK
9098-01	06/14/99	<20	1.50	0.41	0.30	0.47	0.19
9098-02	06/17/99	<20	2.20	0.37	0.29	0.44	0.17
9098-03	06/21/99	<20	1.90	0.41	0.21	0.45	0.21
9098-04	08/25/99	<20	1.80	0.39	0.24	0.46	0.19
9098-05	08/28/99	<20	0.60	0.40	0.21	0.49	0.21
9098-06	09/08/99	<20	1.70	0.39	0.20	0.42	0.20
9098-07	09/12/99	<20	1.60	0.39	0.25	0.48	0.36
9098-08	09/15/99	<20	1.50	0.42	0.20	INDETERMINATE	0.21
9098-09	09/19/99	<20	1.90	0.42	0.23	0.53	0.19
9098-10	10/10/99	<20	3.80	0.41	0.19	0.46	0.23
9098-11	10/12/99	<20	3.10	0.35	0.23	0.44	0.20
9098-12	10/31/99	3588	189.70	4.60	7.43	2.91	7.57
9098-13	11/07/99	27839	1913.60	41.28	78.63	26.6	52.39
9098-14	11/14/99	400582	29000	631.10	983.02	43.64	53.33
9098-15	11/21/99	6614472	150000	7663	6366.28	51.99	53.33
9098-16	11/26/99	89200000	150000	11000	6600.15	38.72	53.33
9098-17	11/28/99	125100000	150000	6418	5236.19	42.73	53.33

PANEL MEMBER	ORTHO VITROS ECI HBsAg	GENERAL BIOLOGICALS SURASE B-96 (TMB) (4SGE3) S/CO	ORTHO VITROS ECI ANTI-HBc TOTAL	ORTHO VITROS ECI ANTI-HBc IgM	ORTHO VITROS ECI ANTI-HBs QUANT	ORTHO VITROS ECI HBeAg	ORTHO VITROS ECI ANTI-HBe
	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK
9098-01	0.04	0.76	3.75	0.01	2.0	0.10	0.22
9098-02	0.14	0.71	3.87	0.01	2.3	0.10	0.23
9098-03	0.05	0.61	3.89	0.01	3.0	0.10	0.23
9098-04	0.04	0.39	3.92	0.01	1.1	0.11	0.23
9098-05	0.04	0.61	3.80	0.01	3.5	0.10	0.24
9098-06	0.04	0.65	4.02	0.01	3.0	0.10	0.23
9098-07	0.04	0.49	4.10	0.01	1.2	0.12	0.23
9098-08	0.04	0.98	3.94	0.01	1.9	0.11	0.23
9098-09	0.04	0.61	4.07	0.01	1.6	0.10	0.22
9089-10	0.03	0.71	4.03	0.01	2.0	0.10	0.21
9089-11	0.04	0.45	4.22	0.01	2.1	0.11	0.22
9089-12	2.54	3.67	4.00	0.01	2.9	0.13	0.23
9089-13	34.3	34.78	4.17	0.01	2.0	0.52	0.21
9089-14	446	81.74	4.35	0.01	2.8	20.5	0.13
9089-15	4540	82.61	4.03	0.01	2.2	620	0.01
9089-16	5540	86.80	3.37	0.02	3.0	2860	0.00
9089-17	4950	87.74	1.89	0.04	2.5	2640	0.00

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for Anti-HIV1/2, HIV p24 Ag, Anti HCV by FDA cleared tests.

This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.

ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
 For Customer Support, please visit www.zeptometrix.com or email custserv@zeptometrix.com