

ZeptoMetrix®

HCV Seroconversion Panel Donor No. 69654 Part Number: HCV10002

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45 u/L)	ALPHA HCV by PCR	ABBOTT MUREX ANTI-HCV 4.0	ABBOTT MUREX HCV Ag/Ab Combo	ABBOTT ANTI-HCV Lab # 1	ABBOTT IMx HCV 3.0 Reagent Pack Lab # 5	ABBOTT AxSYM ANTI-HCV 3.0 Reagent Pack Lab # 5
		TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 8/22/2000	TEST DATE: 7/17/2000
10002-01	11/18/98	9	NEGATIVE	0.221	0.279	0.157	0.150	0.07
10002-02	11/29/98	6	NEGATIVE	0.247	0.276	0.176	0.120	0.06
10002-03	12/07/98	13	POSITIVE	0.287	0.321	0.176	0.160	0.12
10002-04	12/09/98	9	POSITIVE	0.248	6.469	0.157	0.230	0.08
10002-05	12/15/98	28	POSITIVE	0.318	10.062	0.157	0.160	0.08
10002-06	12/17/98	22	POSITIVE	0.286	10.479	0.157	0.140	0.07
10002-07	12/22/98	22	POSITIVE	0.231	11.117	0.157	0.140	0.13

PANEL MEMBER	ABBOTT Architect ANTI-HCV	ABBOTT PRISM ANTI-HCV	ROCHE Cobas Core ANTI-HCV EIA Lab # 5	ORTHO VITROS ANTI-HCV Reagent Pack Lab # 5	ORTHO Enhanced SAVe ANTI-HCV 3.0 Lab # 5	ORTHO ECi HBsAg	ORTHO ECi ANTI-HCV	ORTHO ECi ANTI-HIV ½
	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 9/6/2000	TEST DATE: 8/22/2000	TEST DATE: 7/25/2000	TEST DATE: 11/16/2018	TEST DATE: 11/16/2018	TEST DATE: 11/16/2018
10002-01	0.060	0.087	0.120	0.150	0.00	0.05	0.20	0.07
10002-02	0.060	0.121	0.140	0.120	0.00	0.05	0.22	0.08
10002-03	0.060	0.132	0.150	0.160	0.010	0.06	0.20	0.08
10002-04	0.060	0.183	0.170	0.230	0.020	0.05	0.15	0.08
10002-05	0.060	0.099	0.110	0.160	0.010	0.05	0.16	0.08
10002-06	0.050	0.101	0.120	0.140	0.00	0.05	0.13	0.09
10002-07	0.050	0.096	0.130	0.140	0.010	0.05	0.14	0.08

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHCV10002 Rev03
Effective Date: 10/11/2021