

ZeptoMetrix®

HCV Seroconversion Panel Donor No. 69028 Part Number: HCV10004

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45 u/L)	ALPHA HCV by PCR	ABBOTT MUREX ANTI-HCV 4.0	ABBOTT MUREX HCV Ag/Ab Combo	ABBOTT ANTI-HCV Lab # 1	ABBOTT IMx HCV 3.0 Reagent Pack Lab # 5	ABBOTT AxSYM ANTI-HCV 3.0 Reagent Pack Lab # 5	ABBOTT Architect ANTI-HCV
		TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 8/22/2000	TEST DATE: 7/17/2000	TEST DATE: UNK
10004-01	09/10/98	18	NEGATIVE	0.152	0.293	0.222	0.160	0.120	0.110
10004-02	10/08/98	18	NEGATIVE	0.133	0.286	0.222	0.220	0.140	0.100
10004-03	10/10/98	28	POSITIVE	0.116	0.293	0.200	0.210	0.090	0.110
10004-04	10/15/98	46	POSITIVE	0.133	1.772	0.178	0.230	0.100	0.120
10004-05	10/19/98	38	POSITIVE	0.109	4.983	0.178	0.170	0.120	0.090
10004-06	10/22/98	59	POSITIVE	0.160	8.979	0.133	0.150	0.070	0.110

PANEL MEMBER	ABBOTT PRISM ANTI-HCV	ROCHE Cobas Core ANTI-HCV EIA Lab # 5	ORTHO ECi ANTI-HCV	ORTHO VITROS ANTI-HCV Reagent Pack Lab # 5	ORTHO Enhanced SAVe ANTI-HCV 3.0 Lab # 5	ORTHO ECi HBsAg	ORTHO ECi ANTI-HCV	ORTHO ECi ANTI-HIV ½
	TEST DATE: UNK	TEST DATE: 9/6/2000	TEST DATE: UNK	TEST DATE: 8/22/2000	TEST DATE: 7/25/2000	TEST DATE: 11/16/2018	TEST DATE: 11/16/2018	TEST DATE: 11/16/2018
10004-01	0.129	0.080	0.08	0.160	0.01	0.06	0.05	0.25
10004-02	0.102	0.090	0.07	0.220	0.06	0.06	0.04	0.27
10004-03	0.110	0.080	0.06	0.210	0.01	0.06	0.03	0.26
10004-04	0.113	0.080	0.07	0.230	0.01	0.05	0.05	0.24
10004-05	0.105	0.070	0.05	0.170	0.01	0.05	0.02	0.22
10004-06	0.117	0.070	0.08	0.150	0.00	0.06	0.04	0.18

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHCV10004 Rev03
Effective Date: 10/11/2021