

# ZeptoMetrix®

## HCV Seroconversion Panel Donor No. 66666 Part Number: HCV10011

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility. Donor was un-treated and asymptomatic throughout the collection period.

### HCV Genotype: 1b

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45 u/L)	ALPHA HCV by PCR	ABBOTT MUREX ANTI-HCV 4.0	ABBOTT MUREX HCV Ag/Ab Combo	ABBOTT ANTI-HCV Lab # 1	ABBOTT ANTI-HCV 3.0 Lab # 5	ABBOTT PRISM HCV Lab # 5
		TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 10/25/1999	TEST DATE: 9/21/2000	TEST DATE: 10/4/2000
10011-01	01/08/98	1	NEGATIVE	0.114	0.310	0.158	0.170	0.060
10011-02	01/10/98	1	NEGATIVE	0.107	0.299	0.132	0.100	0.060
10011-03	01/31/98	6	NEGATIVE	0.099	0.306	0.105	0.070	0.060
10011-04	02/02/98	8	NEGATIVE	0.102	0.293	0.132	0.070	0.050
10011-05	02/07/98	7	NEGATIVE	0.101	0.293	0.105	0.070	0.060
10011-06	02/09/98	11	NEGATIVE	0.104	0.303	0.132	0.060	0.050
10011-07	02/14/98	7	NEGATIVE	0.125	0.310	0.132	0.070	0.060
10011-08	02/17/98	4	NEGATIVE	0.119	0.323	0.132	0.070	0.060
10011-09	02/21/98	7	NEGATIVE	0.125	0.296	0.132	0.080	0.080
10011-10	02/24/98	4	NEGATIVE	0.135	0.306	0.158	0.090	0.060
10011-11	02/28/98	7	POSITIVE	0.128	3.769	0.158	0.080	0.070
10011-12	03/05/98	16	POSITIVE	0.131	3.303	0.158	0.100	0.070
10011-13	03/07/98	14	POSITIVE	0.116	2.099	0.132	0.090	0.060

PANEL MEMBER	ABBOTT AxsYM ANTI-HCV 3.0 Reagent Pack Lab # 5	ABBOTT Architect ANTI-HCV	ABBOTT PRISM ANTI-HCV	ROCHE Cobas Core ANTI-HCV EIA	ORTHO Enhanced SAve ANTI-HCV 3.0 Lab # 5	ORTHO Eci HBsAg	ORTHO Eci ANTI-HCV	ORTHO Eci ANTI-HIV ½
	TEST DATE: 7/20/2000	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 9/6/2000	TEST DATE: 9/18/2000	TEST DATE: 11/16/2018	TEST DATE: 11/16/2018	TEST DATE: 11/16/2018
10011-01	0.150	0.100	0.252	0.090	0.000	0.06	0.02	0.12
10011-02	0.090	0.090	0.136	0.060	0.010	0.06	0.01	0.12
10011-03	0.120	0.060	0.105	0.060	0.000	0.05	0.01	0.12
10011-04	0.060	0.060	0.128	0.050	0.000	0.05	0.01	0.12
10011-05	0.110	0.050	0.147	0.050	0.000	0.05	0.00	0.12
10011-06	0.070	0.060	0.098	0.050	0.000	0.05	0.01	0.11
10011-07	0.090	0.070	0.119	0.050	0.000	0.05	0.01	0.12
10011-08	0.080	0.070	0.136	0.060	0.000	0.04	0.01	0.12
10011-09	0.090	0.070	0.135	0.060	0.000	0.05	0.02	0.11
10011-10	0.160	0.110	0.096	0.060	0.000	0.05	0.02	0.11
10011-11	0.090	0.070	0.262	0.060	0.000	0.05	0.02	0.11
10011-12	0.140	0.070	0.092	0.060	0.000	0.05	0.02	0.11
10011-13	0.100	0.060	0.101	0.060	0.010	0.04	0.01	0.13

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)

PIHCV10011 Rev04  
Effective Date: 10/11/2021