

# ZeptoMetrix®

## HCV Seroconversion Panel Donor No. 66747 Part Number: HCV10012

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility. Donor was un-treated and asymptomatic throughout the collection period.

### HCV Genotype: 1a

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45 u/L)	ALPHA HCV by PCR	ABBOTT MUREX ANTI-HCV 4.0	ABBOTT MUREX HCV Ag/Ab Combo	ABBOTT ANTI-HCV Lab # 1	ABBOTT AxSYM ANTI-HCV 3.0 Reagent Pack Lab # 5
		TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 10/25/1999	TEST DATE: 9/19/2000
10012-01	02/22/98	11	NEGATIVE	0.137	0.310	0.158	0.150
10012-02	02/24/98	14	NEGATIVE	0.152	0.320	0.132	0.190
10012-03	03/01/98	18	NEGATIVE	0.146	0.337	0.158	0.170
10012-04	03/03/98	26	NEGATIVE	0.147	0.296	0.158	0.160
10012-05	03/08/98	14	POSITIVE	0.122	0.289	0.158	0.240
10012-06	03/10/98	12	POSITIVE	0.122	0.323	0.105	0.160
10012-07	03/15/98	23	POSITIVE	0.135	0.398	0.132	0.130
10012-08	03/17/98	16	POSITIVE	0.125	0.490	0.132	0.220
10012-09	03/22/98	21	POSITIVE	0.144	0.490	0.105	0.170
10012-10	03/24/98	16	POSITIVE	0.138	0.486	0.105	0.180
10012-11	03/29/98	22	POSITIVE	0.117	0.503	0.105	0.150
10012-12	03/31/98	36	POSITIVE	0.111	0.435	0.105	0.120
10012-13	04/05/98	73	POSITIVE	0.117	0.449	0.132	0.110

PANEL MEMBER	ABBOTT Architect ANTI-HCV	ABBOTT PRISM ANTI-HCV	ORTHO Enhanced SAve ANTI-HCV 3.0 Lab # 5	ORTHO ECi HBsAg	ORTHO ECi ANTI-HCV	ORTHO ECi ANTI-HIV ½
	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 9/19/2000	TEST DATE: 11/16/2018	TEST DATE: 11/16/2018	TEST DATE: 11/16/2018
10012-01	0.060	0.300	0.010	0.06	0.02	0.16
10012-02	0.060	0.265	0.010	0.06	0.02	0.15
10012-03	0.060	0.300	0.010	0.05	0.02	0.15
10012-04	0.060	0.329	0.000	0.05	0.01	0.15
10012-05	0.050	0.297	0.010	0.05	0.01	0.15
10012-06	0.080	0.336	0.000	0.05	0.02	0.14
10012-07	0.070	0.302	0.000	0.06	0.03	0.15
10012-08	0.040	0.274	0.000	0.05	0.01	0.14
10012-09	0.210	0.314	0.010	0.06	0.02	0.14
10012-10	0.040	0.264	0.000	0.04	0.01	0.14
10012-11	0.060	0.277	0.010	0.05	0.01	0.16
10012-12	0.040	0.294	0.040	0.06	0.01	0.17
10012-13	0.040	0.258	0.050	0.06	0.02	0.19

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
 This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
 For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)

PIHCV10012 Rev04  
 Effective Date: 10/11/2021