

# ZeptoMetrix®

## HCV Seroconversion Panel Donor No. 67356 Part Number: HCV10013

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

### HCV Genotype: 1a

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45 u/L)	ALPHA HCV by PCR	ABBOTT MUREX ANTI-HCV 4.0	ABBOTT MUREX HCV Ag/Ab Combo	ABBOTT ANTI-HCV Lab # 1	ABBOTT AxSYM ANTI-HCV 3.0 Reagent Pack Lab # 5
		TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 10/25/1999	TEST DATE: 9/19/2000
10013-01	03/31/98	8	NEGATIVE	0.089	0.293	0.079	0.090
10013-02	04/09/98	8	NEGATIVE	0.092	0.286	0.079	0.070
10013-03	04/12/98	8	NEGATIVE	0.086	0.299	0.079	0.100
10013-04	05/16/98	8	POSITIVE	0.102	2.993	0.105	0.100
10013-05	05/19/98	5	POSITIVE	0.134	9.929	0.105	0.140
10013-06	05/23/98	4	POSITIVE	0.207	3.303	0.105	0.080
10013-07	05/26/98	23	POSITIVE	0.565	4.027	0.105	0.130
10013-08	05/30/98	188	POSITIVE	1.234	3.473	0.132	0.140
10013-09	06/03/98	212	POSITIVE	2.027	3.248	0.211	0.180
10013-10	06/06/98	328	POSITIVE	2.928	2.922	0.237	0.330

PANEL MEMBER	ABBOTT Architect ANTI-HCV	ABBOTT PRISM ANTI-HCV	ORTHO Enhanced SAve ANTI-HCV 3.0 Lab # 5	ORTHO ECi HBsAg	ORTHO ECi ANTI-HCV	ORTHO ECi ANTI-HIV ½
	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 9/19/2000	TEST DATE: 11/16/2018	TEST DATE: 11/16/2018	TEST DATE: 11/16/2018
10013-01	0.030	0.123	0.000	0.06	0.01	0.15
10013-02	0.030	0.098	0.010	0.16	0.01	0.15
10013-03	0.020	0.088	0.010	0.16	0.01	0.15
10013-04	0.050	0.108	0.000	0.18	0.02	0.17
10013-05	0.040	0.093	0.000	0.20	0.02	0.14
10013-06	0.030	0.102	0.010	0.18	0.02	0.14
10013-07	0.040	0.091	0.010	0.15	0.06	0.14
10013-08	0.050	0.108	0.010	0.15	0.22	0.14
10013-09	0.120	0.110	0.020	0.14	0.77	0.13
10013-10	0.330	0.276	0.030	0.14	1.58	0.14

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)

PIHCV10013 Rev04  
Effective Date: 10/12/2021