



**HCV Seroconversion Panel Donor No. 68744  
Part Number: HCV10020**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

**HCV Genotype: 1a**

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45 u/L)	ALPHA HCV by PCR	ABBOTT MUREX ANTI-HCV 4.0	ABBOTT MUREX HCV Ag/Ab Combo	ABBOTT ANTI-HCV Lab # 1	ABBOTT ANTI-HCV 3.0 Lab #5
		TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 10/27/1999	TEST DATE: 7/31/2000
10020-01	08/27/98	2	NEGATIVE	0.152	0.342	0.213	0.370
10020-02	08/29/98	1	NEGATIVE	0.173	0.369	0.230	0.280
10020-03	09/08/98	6	NEGATIVE	0.153	0.434	0.213	0.200
10020-04	09/11/98	7	NEGATIVE	0.153	0.542	0.246	0.270
10020-05	09/16/98	4	NEGATIVE	0.182	0.783	0.230	0.330
10020-06	09/20/98	4	POSITIVE	0.129	1.569	0.197	0.300
10020-07	09/25/98	10	POSITIVE	0.120	1.366	0.230	0.270
10020-08	09/27/98	5	POSITIVE	0.111	4.797	0.230	0.260
10020-09	10/08/98	9	POSITIVE	0.132	9.369	0.230	0.130
10020-10	10/12/98	7	POSITIVE	0.130	7.153	0.213	0.170
10020-11	10/15/98	9	POSITIVE	0.136	10.078	0.197	0.290
10020-12	10/19/98	6	POSITIVE	0.144	11.041	0.246	0.190
10020-13	10/22/98	8	POSITIVE	0.123	8.010	0.230	0.760

PANEL MEMBER	ABBOTT AxSYM ANTI-HCV 3.0 Reagent Pack Lab # 5	ABBOTT Architect ANTI-HCV	ABBOTT PRISM ANTI-HCV	SANOFI Monolisa ANTI-HCV PLUS Lab # 5	ORTHO Eci HBsAg	ORTHO Eci ANTI-HCV	ORTHO Eci ANTI-HIV 1/2
	TEST DATE: 8/10/2000	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 10/16/2000	TEST DATE: 11/16/2018	TEST DATE: 11/16/2018	TEST DATE: 11/16/2018
10020-01	0.110	0.090	0.123	0.000	0.06	0.02	0.11
10020-02	0.140	0.080	0.096	0.000	0.06	0.02	0.12
10020-03	0.150	0.080	0.091	0.000	0.06	0.02	0.11
10020-04	0.150	0.080	0.095	0.000	0.05	0.02	0.13
10020-05	0.130	0.080	0.095	0.000	0.05	0.01	0.13
10020-06	0.090	0.070	0.093	0.000	0.05	0.01	0.13
10020-07	0.130	0.070	0.138	0.000	0.06	0.02	0.14
10020-08	0.130	0.060	0.101	0.000	0.06	0.01	0.13
10020-09	0.180	0.070	0.308	0.000	0.06	0.01	0.14
10020-10	0.150	0.080	0.100	0.000	0.05	0.01	0.15
10020-11	0.160	0.080	0.156	0.000	0.06	0.01	0.13
10020-12	0.150	0.070	0.104	0.000	0.06	0.01	0.12
10020-13	0.110	0.070	0.125	0.000	0.06	0.01	0.13

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

**This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.**

ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [custserv@zeptometrix.com](mailto:custserv@zeptometrix.com)