



**HCV Seroconversion Panel Donor No. 68450  
Part Number: HCV10022**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

**HCV Genotype: 1a**

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45 u/L)  TEST DATE: UNK	ALPHA HCV by PCR  TEST DATE: UNK	ABBOTT MUREX ANTI-HCV 4.0  TEST DATE: UNK	ABBOTT MUREX HCV Ag/Ab Combo  TEST DATE: UNK	ABBOTT ANTI-HCV Lab # 1  TEST DATE: 11/27/1999	ABBOTT ANT-HCV 3.0 Lab # 5  TEST DATE: 7/31/2000
10022-01	08/20/98	3	NEGATIVE	0.166	0.366	0.180	0.240
10022-02	08/22/98	1	NEGATIVE	0.135	0.369	0.213	0.180
10022-03	08/27/98	0	NEGATIVE	0.144	0.383	0.180	0.230
10022-04	08/29/98	0	NEGATIVE	0.145	0.372	0.164	0.150
10022-05	09/03/98	4	NEGATIVE	0.138	0.359	0.148	0.160
10022-06	09/09/98	8	POSITIVE	0.136	1.197	0.197	0.170
10022-07	09/12/98	11	POSITIVE	0.132	0.348	0.148	0.170
10022-08	09/17/98	18	POSITIVE	0.144	0.469	0.180	0.180
10022-09	09/20/98	16	POSITIVE	0.127	0.434	0.180	0.180
10022-10	09/25/98	11	POSITIVE	0.113	0.431	0.148	0.140
10022-11	09/27/98	7	POSITIVE	0.115	0.428	0.148	0.160

PANEL MEMBER	ABBOTT AxSYM ANTI-HCV 3.0 Reagent Pack Lab # 5  TEST DATE: 8/10/2000	ABBOTT Architect ANTI-HCV  TEST DATE: UNK	ABBOTT PRISM ANTI-HCV  TEST DATE: UNK	ORTHO Enhanced SAVe ANTI-HCV 3.0 Lab # 5  TEST DATE: 10/16/2000	ORTHO ECi HBsAg  TEST DATE: 11/16/2018	ORTHO ECi ANTI-HCV  TEST DATE: 11/16/2018	ORTHO ECi ANTI-HIV ½  TEST DATE: 11/16/2018
10022-01	0.230	0.100	0.451	0.000	0.06	0.04	0.11
10022-02	0.170	0.090	0.458	0.000	0.05	0.03	0.11
10022-03	0.190	0.090	0.471	0.000	0.05	0.03	0.10
10022-04	0.120	0.090	0.434	0.000	0.06	0.03	0.11
10022-05	0.160	0.090	0.440	0.000	0.06	0.02	0.12
10022-06	0.120	0.080	0.441	0.000	0.06	0.02	0.10
10022-07	0.190	0.070	0.426	0.000	0.06	0.02	0.11
10022-08	0.200	0.080	0.390	0.000	0.07	0.02	0.14
10022-09	0.200	0.090	0.422	0.000	0.06	0.02	0.10
10022-10	0.180	0.080	0.382	0.000	0.06	0.01	0.10
10022-11	0.120	0.070	0.409	0.000	0.05	0.01	0.10

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)

PIHCV10022 Rev04  
Effective Date: 10/12/2021