



HCV Seroconversion Panel Donor No. 67505
Part Number: HCV10023

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

HCV Genotype: 1a

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45 u/L)	ALPHA HCV by PCR	ABBOTT MUREX ANTI-HCV 4.0	ABBOTT MUREX HCV Ag/Ab Combo	ABBOTT ANTI-HCV Lab # 1	ABBOTT AxSYM ANTI-HCV3.0 Reagent Pack Lab # 5
		TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK
10023-01	04/11/98	10	NEGATIVE	0.159	0.356	0.189	0.150
10023-02	04/14/98	7	NEGATIVE	0.126	0.346	0.135	0.150
10023-03	04/18/98	7	NEGATIVE	0.124	0.319	0.135	0.160
10023-04	04/21/98	6	NEGATIVE	0.100	0.336	0.162	0.130
10023-05	04/25/98	3	NEGATIVE	0.123	0.312	0.162	0.170
10023-06	04/29/98	4	NEGATIVE	0.115	0.319	0.162	0.080
10023-07	05/02/98	19	NEGATIVE	0.136	0.329	0.135	0.130
10023-08	05/06/98	7	NEGATIVE	0.127	0.336	0.135	0.150
10023-09	05/09/98	10	NEGATIVE	0.161	0.346	0.135	0.140
10023-10	05/13/98	8	NEGATIVE	0.138	0.353	0.135	0.300
10023-11	05/16/98	9	NEGATIVE	0.136	0.342	0.216	0.100
10023-12	05/20/98	2	POSITIVE	0.106	0.339	0.135	0.110
10023-13	05/23/98	6	POSITIVE	0.102	3.966	0.135	0.120
10023-14	05/27/98	14	POSITIVE	0.111	8.119	0.081	0.150
10023-15	05/30/98	8	POSITIVE	0.120	11.546	0.162	0.120
10023-16	06/03/98	15	POSITIVE	0.136	11.749	0.189	0.120
10023-17	06/06/98	17	POSITIVE	0.129	11.464	0.162	0.100
10023-18	06/10/98	15	POSITIVE	0.145	11.675	0.135	0.140
10023-19	06/13/98	8	POSITIVE	0.117	11.631	0.108	0.100
10023-20	06/17/98	10	POSITIVE	0.124	10.773	0.162	0.100
10023-21	06/21/98	31	POSITIVE	0.100	10.075	0.162	0.130

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHCV10023 Rev04
Effective Date: 10/12/2021



HCV Seroconversion Panel Donor No. 67505
Part Number: HCV10023

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	ABBOTT Architect ANTI-HCV	ABBOTT PRISM ANTI-HCV	SANOFI Monolisa ANTI-HCV PLUS Lab # 5	ORTHO ECi HBsAg	ORTHO ECi ANTI-HCV	ORTHO ECi ANTI-HIV ½
	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 10/20/2000	TEST DATE: 11/16/2018	TEST DATE: 11/16/2018	TEST DATE: 11/16/2018
10023-01	0.130	0.099	0.200	0.07	0.03	0.12
10023-02	0.100	0.116	0.200	0.07	0.02	0.12
10023-03	0.080	0.082	0.190	0.06	0.02	0.12
10023-04	0.100	0.116	0.180	0.06	0.02	0.12
10023-05	0.090	0.083	0.170	0.07	0.02	0.11
10023-06	0.080	0.116	0.180	0.06	0.01	0.11
10023-07	0.090	0.109	0.180	0.06	0.02	0.11
10023-08	0.090	0.096	0.180	0.06	0.02	0.12
10023-09	0.110	0.084	0.190	0.08	0.03	0.12
10023-10	0.100	0.124	0.180	0.08	0.02	0.12
10023-11	0.100	0.087	0.180	0.07	0.01	0.13
10023-12	0.090	0.100	0.180	0.06	0.01	0.12
10023-13	0.080	0.113	0.180	0.06	0.02	0.11
10023-14	0.090	0.134	0.180	0.06	0.01	0.11
10023-15	0.100	0.113	0.190	0.08	0.02	0.12
10023-16	0.080	0.313	0.190	0.07	0.03	0.11
10023-17	0.070	0.093	0.180	0.06	0.02	0.11
10023-18	0.070	0.098	0.170	0.06	0.02	0.11
10023-19	0.070	0.106	0.170	0.06	0.01	0.11
10023-20	0.070	0.099	0.170	0.07	0.02	0.10
10023-21	0.070	0.119	0.170	0.05	0.02	0.12

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com