



HCV Seroconversion Panel Donor No. 66719
Part Number: HCV10024

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
 Donor was un-treated and asymptomatic throughout the collection period.

HCV Genotype: 1a

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45 u/L)	ALPHA HCV by PCR	ABBOTT MUREX ANTI-HCV 4.0	ABBOTT MUREX HCV Ag/Ab Combo	ABBOTT ANTI-HCV Lab # 1	ABBOTT AxSYM ANTI-HCV 3.0 Reagent Pack Lab # 5
		TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 10/25/1999	TEST DATE: UNK
10024-01	01/21/98	0	NEGATIVE	0.100	0.315	0.243	0.210
10024-02	01/23/98	2	NEGATIVE	0.091	0.353	0.243	0.160
10024-03	01/28/98	8	NOT TESTED	0.091	0.319	0.243	0.190
10024-04	02/03/98	9	NEGATIVE	0.091	0.349	0.243	0.170
10024-05	02/25/98	5	NEGATIVE	0.111	0.363	0.243	0.170
10024-06	03/12/98	5	POSITIVE	0.092	1.153	0.243	0.170
10024-07	03/16/98	9	POSITIVE	0.086	11.186	0.270	0.200
10024-08	03/18/98	10	POSITIVE	0.082	11.417	0.162	0.210

PANEL MEMBER	ABBOTT Architect ANTI-HCV	ABBOTT PRISM ANTI-HCV	SANOVI Monolisa ANTI-HCV PLUS Lab # 5	ORTHO ECi HBsAg	ORTHO ECi ANTI-HCV	ORTHO ECi ANTI-HIV 1/2
	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 10/20/2000	TEST DATE: 11/16/2018	TEST DATE: 11/16/2018	TEST DATE: 11/16/2018
10024-01	0.080	0.092	0.170	0.06	0.01	0.11
10024-02	0.070	0.082	0.170	0.06	0.01	0.11
10024-03	0.060	0.083	0.160	0.07	0.01	0.10
10024-04	0.070	0.095	0.180	0.07	0.01	0.11
10024-05	0.080	0.086	0.170	0.08	0.02	0.13
10024-06	0.070	0.087	0.190	0.07	0.01	0.14
10024-07	0.080	0.081	0.180	0.06	0.01	0.13
10024-08	0.060	0.087	0.170	0.06	0.01	0.14

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
 This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
 For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com