



**HCV Seroconversion Panel Donor No. 66751
Part Number: HCV10027**

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

HCV Genotype: 1b

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45 u/L)	ALPHA HCV by PCR	ABBOTT MUREX ANTI-HCV 4.0	ABBOTT MUREX HCV Ag/Ab Combo	ABBOTT ANTI-HCV Lab # 1	ABBOTT IMx HCV 3.0 Reagent Pack Lab # 5
		TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 10/25/1999	TEST DATE: 10/17/2000
10027-01	01/30/98	4	NOT TESTED	0.208	0.481	0.135	0.150
10027-02	02/10/98	11	NEGATIVE	0.121	0.383	0.216	0.100
10027-03	02/12/98	5	NEGATIVE	0.205	0.390	0.189	0.130
10027-04	02/17/98	3	NEGATIVE	0.208	0.400	0.135	0.160
10027-05	02/23/98	9	NEGATIVE	0.217	0.424	0.162	0.120
10027-06	02/26/98	22	NEGATIVE	0.127	0.346	0.162	0.110
10027-07	03/03/98	55	POSITIVE	0.153	1.186	0.189	0.130
10027-08	03/09/98	67	POSITIVE	0.245	1.393	0.216	0.070

PANEL MEMBER	ABBOTT Architect ANTI-HCV	ABBOTT PRISM ANTI-HCV	ORTHO Enhanced SAve ANTI-HCV 3.0 Lab # 5	ORTHO Eci HBsAg	ORTHO Eci ANTI-HCV	ORTHO Eci ANTI-HIV 1/2
	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 9/29/2000	TEST DATE: 11/16/2018	TEST DATE: 11/16/2018	TEST DATE: 11/16/2018
10027-01	0.070	0.105	0.010	0.07	0.02	0.11
10027-02	0.060	0.118	0.090	0.07	0.05	0.12
10027-03	0.060	0.131	0.090	0.09	0.13	0.10
10027-04	0.160	0.088	0.030	0.09	0.13	0.10
10027-05	0.100	0.129	0.060	0.08	0.08	0.10
10027-06	0.050	0.095	0.020	0.07	0.02	0.11
10027-07	0.070	0.099	0.030	0.07	0.02	0.11
10027-08	0.170	0.106	0.000	0.08	0.05	0.10

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com