



**HCV Seroconversion Panel Donor No. 65977
Part Number: HCV10030**

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

HCV Genotype: 1a

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45 u/L)	ALPHA HCV by PCR	ABBOTT MUREX ANTI-HCV 4.0	ABBOTT MUREX HCV Ag/Ab Combo	ABBOTT ANTI-HCV Lab # 1	ABBOTT ANTI-HCV 3.0 Lab # 5
		TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 10/25/1999	TEST DATE: 10/16/2000
10030-01	12/08/97	10	NEGATIVE	0.127	0.376	0.162	0.100
10030-02	12/10/97	10	NEGATIVE	0.120	0.332	0.135	0.110
10030-03	12/17/97	16	NEGATIVE	0.114	0.366	0.108	0.090
10030-04	12/19/97	8	NEGATIVE	0.108	0.329	0.108	0.110
10030-05	12/24/97	6	NEGATIVE	0.114	0.339	0.135	0.080
10030-06	12/26/97	11	NEGATIVE	0.108	0.359	0.108	0.090
10030-07	12/31/97	12	POSITIVE	0.115	0.363	0.162	0.080
10030-08	01/02/98	8	POSITIVE	0.123	0.647	0.135	0.070

PANEL MEMBER	ABBOTT IMx HCV 3.0 Reagent Pack Lab # 5	ABBOTT Architect ANTI-HCV	ABBOTT PRISM ANTI-HCV	ORTHO Enhanced SAve ANTI-HCV 3.0 Lab # 5	ORTHO ECi HBsAg	ORTHO ECi ANTI-HCV	ORTHO ECi ANTI-HIV 1/2
	TEST DATE: 10/17/2000	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 9/29/2000	TEST DATE: 11/16/2018	TEST DATE: 11/16/2018	TEST DATE: 11/16/2018
10030-01	0.100	0.050	0.096	0.000	0.07	0.02	0.11
10030-02	0.160	0.050	0.081	0.000	0.07	0.01	0.10
10030-03	0.100	0.050	0.089	0.000	0.07	0.01	0.10
10030-04	0.130	0.050	0.073	0.000	0.07	0.01	0.10
10030-05	0.090	0.040	0.073	0.000	0.07	0.01	0.10
10030-06	0.120	0.030	0.069	0.000	0.06	0.01	0.09
10030-07	0.120	0.050	0.078	0.000	0.06	0.01	0.10
10030-08	0.090	0.050	0.071	0.000	0.06	0.01	0.09

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com