



**HCV Seroconversion Panel Donor No.68223
Part Number: HCV10036**

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

HCV Genotype: 1b

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45 u/L)	ALPHA HCV by PCR	ABBOTT MUREX ANTI-HCV 4.0	ABBOTT MUREX HCV Ag/Ab Combo	ABBOTT ANTI-HCV Lab # 1	ABBOTT Architect ANTI-HCV
		TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 10/28/1999	TEST DATE: UNK
10036-01	07/15/98	13	NEGATIVE	0.104	0.284	0.098	0.050
10036-02	07/22/98	21	NEGATIVE	0.104	0.315	0.137	0.060
10036-03	07/24/98	9	NEGATIVE	0.095	0.308	0.118	0.050
10036-04	08/12/98	10	POSITIVE	0.133	0.695	0.157	0.060
10036-05	08/14/98	18	POSITIVE	0.102	0.336	0.137	0.050
10036-06	08/19/98	20	POSITIVE	0.111	0.414	0.137	0.050
10036-07	08/21/98	11	POSITIVE	0.105	0.339	0.118	0.040
10036-08	08/26/98	8	POSITIVE	0.111	0.414	0.118	0.060
10036-09	08/28/98	12	POSITIVE	0.114	0.377	0.137	0.060
10036-10	09/03/98	36	POSITIVE	0.107	0.312	0.137	0.050
10036-11	09/05/98	26	POSITIVE	0.105	0.349	0.118	0.050
10036-12	09/10/98	63	POSITIVE	0.105	0.377	0.137	0.050
10036-13	09/13/98	46	POSITIVE	0.099	0.322	0.098	0.040

PANEL MEMBER	ABBOTT PRISM ANTI-HCV	ORTHO HCV 3.0 ELISA Testsystem Enhanced SAVe	ROCHE Cobas Core ANTI-HCV EIA	ORTHO ECi HBsAg	ORTHO ECi ANTI-HCV	ORTHO ECi ANTI-HIV 1/2
	TEST DATE: UNK	TEST DATE: 10/23/2000	TEST DATE: 9/7/2000	TEST DATE: 11/16/2018	TEST DATE: 11/16/2018	TEST DATE: 11/16/2018
10036-01	0.101	0.010	0.000	0.06	0.01	0.08
10036-02	0.109	0.000	0.100	0.08	0.01	0.09
10036-03	0.083	0.010	0.100	0.07	0.01	0.09
10036-04	0.094	0.000	0.100	0.09	0.01	0.09
10036-05	0.098	0.000	0.100	0.07	0.01	0.08
10036-06	0.130	0.000	0.100	0.08	0.01	0.09
10036-07	0.092	0.000	0.100	0.07	0.01	0.09
10036-08	0.104	0.000	0.100	0.07	0.01	0.09
10036-09	0.101	0.010	0.100	0.80	0.01	0.08
10036-10	0.107	0.000	0.100	0.06	0.01	0.09
10036-11	0.095	0.000	0.100	0.07	0.01	0.08
10036-12	0.095	0.000	0.100	0.07	0.01	0.08
10036-13	0.098	0.000	0.100	0.07	0.01	0.09

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.

ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
For Customer Support, please visit www.zeptometrix.com or email custserv@zeptometrix.com