

ZeptoMetrix®

HCV Seroconversion Panel Donor No.65643 Part Number: HCV10043

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

HCV Genotype: 1

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45 u/L)	ALPHA HCV by PCR	ABBOTT MUREX ANTI-HCV 4.0	ABBOTT MUREX HCV Ag/Ab Combo	ABBOTT ANTI-HCV Lab # 1	ABBOTT ANTI-HCV 3.0 Lab # 5	ABBOTT IMxIMx HCV 3.0 Reagent Pack Lab # 5
		TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 10/28/1999	TEST DATE: 7/4/2000	TEST DATE: 7/25/2000
10043-01	11/09/97	24	POSITIVE	0.110	0.565	0.118	0.100	0.140
10043-02	11/18/97	10	POSITIVE	0.120	0.747	0.137	0.120	0.120
10043-03	12/04/97	28	POSITIVE	0.108	0.428	0.137	0.120	0.120
10043-04	12/08/97	19	POSITIVE	0.123	0.938	0.137	0.120	0.090
10043-05	12/11/97	16	NOT TESTED	0.120	0.856	0.137	0.120	0.100
10043-06	12/15/97	27	POSITIVE	0.108	0.764	0.176	0.130	0.090
10043-07	12/18/97	34	POSITIVE	0.108	0.634	0.137	0.110	0.100
10043-08	12/22/97	51	POSITIVE	0.114	0.836	0.137	0.110	0.090
10043-09	12/26/97	63	POSITIVE	0.131	0.771	0.157	0.130	0.130
10043-10	12/29/97	84	POSITIVE	0.163	1.171	0.176	0.110	0.230

PANEL MEMBER	ABBOTT AxSYM ANTI-HCV 3.0 Reagent Pack Lab # 5	ABBOTT ARCHITECT ANTI-HCV	ABBOTT PRISM ANTI-HCV	ROCHE Cobas Core ANTI-HCV EIA	ORTHO Enhanced SAve ANTI-HCV 3.0 Lab # 5	ORTHO VITROS ECi ANTI-HCV	SANOFI Monolisa ANTI-HCV PLUS Lab # 5
	TEST DATE: 9/1/2000	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 9/6/2000	TEST DATE: 9/18/2000	TEST DATE: UNK	TEST DATE: 8/28/2000
10043-01	0.170	0.070	0.124	0.070	0.010	0.04	0.070
10043-02	0.100	0.070	0.138	0.070	0.000	0.04	0.100
10043-03	0.160	0.070	0.114	0.060	0.010	0.03	0.090
10043-04	0.130	0.070	0.139	0.060	0.010	0.04	0.090
10043-05	0.150	0.080	0.145	0.070	0.000	0.04	0.090
10043-06	0.140	0.080	0.139	0.070	0.000	0.04	0.080
10043-07	0.150	0.070	0.109	0.060	0.030	0.03	0.090
10043-08	0.150	0.070	0.116	0.070	0.000	0.04	0.060
10043-09	0.240	0.110	0.142	0.060	0.000	0.05	0.080
10043-10	0.300	0.260	0.237	0.070	0.020	0.10	0.070

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHCV10043 Rev04
Effective Date: 10/14/2021