



**HCV Seroconversion Panel Donor No.65982
Part Number: HCV10044**

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

HCV Genotype: 1a

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45 u/L) TEST DATE: UNK	ALPHA HCV by PCR TEST DATE: UNK	ABBOTT MUREX ANTI-HCV 4.0 TEST DATE: UNK	ABBOTT MUREX HCV Ag/Ab Combo TEST DATE: UNK	ABBOTT ANTI-HCV Lab # 1 TEST DATE: 10/28/1999	ABBOTT ANTI-HCV 3.0 Lab # 5 TEST DATE: 10/16/2000
10044-01	01/06/98	28	POSITIVE	0.143	2.045	0.176	0.140
10044-02	01/08/98	28	POSITIVE	0.122	0.675	0.176	0.120
10044-03	01/13/98	47	POSITIVE	0.127	1.113	0.176	0.130
10044-04	01/20/98	28	POSITIVE	0.113	2.003	0.157	0.110

PANEL MEMBER	ABBOTT IMx HCV 3.0 Reagent Pack Lab # 5 TEST DATE: 10/17/2000	ABBOTT Architect ANTI-HCV TEST DATE: UNK	ABBOTT PRISM ANTI-HCV TEST DATE: UNK	ORTHO Enhanced SAVe ANTI-HCV 3.0 Lab # 5 TEST DATE: 9/29/2000	ORTHO ECi HBsAg TEST DATE: 11/16/2018	ORTHO ECi ANTI-HCV TEST DATE: 11/16/2018	ORTHO ECi ANTI-HIV ½ TEST DATE: 11/16/2018
10044-01	0.240	0.100	0.138	0.010	0.08	0.03	0.09
10044-02	0.180	0.090	0.129	0.000	0.07	0.02	0.10
10044-03	0.260	0.080	0.139	0.000	0.08	0.02	0.09
10044-04	0.150	0.080	0.160	0.000	0.08	0.01	0.11

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHCV10044 Rev04
Effective Date: 10/14/2021