



**HCV Seroconversion Panel Donor No.420245049
Part Number: HCV10057**

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility. Donor was un-treated and asymptomatic throughout the collection period.

HCV Genotype: 1a

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45 u/L)	VENDOR ANTI-HCV Ab	ABBOTT MUREX ANTI-HCV 4.0	ABBOTT MUREX HCV Ag/Ab Combo	ABBOTT Architect ANTI-HCV
		TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK
10057-01	08/24/98	13	NEGATIVE	0.103	0.286	0.050
10057-02	09/15/98	13	NEGATIVE	0.095	0.293	0.040
10057-03	09/21/98	11	NEGATIVE	0.095	0.282	0.050
10057-04	09/28/98	11	NEGATIVE	0.078	0.296	0.030
10057-05	09/30/98	13	NEGATIVE	0.094	0.310	0.040
10057-06	01/23/99	170	POSITIVE	5.419	9.951	9.950
10057-07	01/25/99	87	POSITIVE	5.296	9.819	11.050

PANEL MEMBER	ABBOTT PRISM ANTI-HCV	ABBOTT Ab	ORTHO Ab 3.0	ABBOTT ANTI-HCV Lab #1	ORTHO ECi HBsAg	ORTHO ECi ANTI-HCV	ORTHO ECi ANTI-HIV ½
	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 3/24/2000	TEST DATE: 11/23/1999	TEST DATE: 11/16/2018	TEST DATE: 11/16/2018	TEST DATE: 11/16/2018
10057-01	0.088	0.09	0.196	0.196	0.06	0.01	0.10
10057-02	0.095	0.15	0.174	0.174	0.07	0.01	0.10
10057-03	0.086	0.14	0.174	0.174	0.07	0.01	0.10
10057-04	0.085	0.13	0.130	0.130	0.06	0.00	0.11
10057-05	0.092	0.12	0.196	0.196	0.07	0.01	0.11
10057-06	8.119	5.83	4.109	4.109	0.06	17.6	0.14
10057-07	8.115	5.39	3.891	3.891	0.07	17.6	0.12

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com