

# ZeptoMetrix®

## HCV Seroconversion Panel Donor No.90163052 Part Number: HCV10071

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

### HCV Genotype: 1a

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45 u/L)	VENDOR ANTI-HCV Ab	ABBOTT MUREX ANTI-HCV 4.0	ABBOTT MUREX HCV Ag/Ab Combo	VIROMED PCR	ABBOTT PRISM ANTI-HCV	ABBOTT Architect ANTI-HCV
		TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 1/26/2000	TEST DATE: UNK	TEST DATE: UNK
10071-01	02/27/98	18	NEGATIVE	0.103	0.314	NEGATIVE	0.082	0.030
10071-02	05/13/98	38	NEGATIVE	0.297	0.850	7,200 COPIES	1.498	0.740
10071-03	05/15/98	54	NEGATIVE	0.415	1.850	9,790 COPIES	2.559	1.590
10071-04	05/20/98	67	NEGATIVE	1.653	7.007	NEGATIVE	5.869	6.890
10071-05	05/22/98	52	POSITIVE	1.896	7.307	3,110 COPIES	6.504	8.680
10071-06	05/27/98	53	POSITIVE	3.952	10.460	4,030 COPIES	8.071	10.820
10071-07	05/29/98	35	POSITIVE	4.825	11.659	5,750 COPIES	7.619	12.110

PANEL MEMBER	EY LABS HCVSCAN	CHIRON PCR	ABBOTT ANTI-HCV AbAb	ORTHO HCV AbAb	VENDOR HBsAg	VENDOR HIV p24 Ag
	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 11/23/1999	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK
10071-01	NEGATIVE	<0.2	0.130	0.030	NEGATIVE	NEGATIVE
10071-02	POSITIVE	<0.2	0.239	0.913	NEGATIVE	NEGATIVE
10071-03	POSITIVE	<0.2	0.522	2.116	NEGATIVE	NEGATIVE
10071-04	POSITIVE	<0.2	1.413	4.871	NEGATIVE	NEGATIVE
10071-05	POSITIVE	<0.2	2.174	4.871	NEGATIVE	NEGATIVE
10071-06	POSITIVE	<0.2	2.848	4.871	NEGATIVE	NEGATIVE
10071-07	POSITIVE	<0.2	3.565	4.871	NEGATIVE	NEGATIVE

PANEL MEMBER	VENDOR ANTI-HIV ½	VENDOR HCV PC	ORTHO ECi HBsAg	ORTHO ECi ANTI-HCV	ORTHO ECi ANTI-HIV ½
	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 11/16/2018	TEST DATE: 11/16/2018	TEST DATE: 11/16/2018
10071-01	NEGATIVE	NOT TESTED	0.05	0.01	0.11
10071-02	NEGATIVE	NOT TESTED	0.06	0.34	0.11
10071-03	NEGATIVE	NOT TESTED	0.07	1.26	0.11
10071-04	NEGATIVE	NOT TESTED	0.11	5.66	0.12
10071-05	NEGATIVE	POSITIVE	0.11	7.41	0.13
10071-06	NEGATIVE	NOT TESTED	0.12	9.87	0.12
10071-07	NEGATIVE	NOT TESTED	0.18	10.1	0.11

#### Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)

PIHCV10071 Rev04  
Effective Date: 10/15/2021