

ZeptoMetrix®

HCV Seroconversion Panel Donor No.66822 Part Number: HCV10235

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45 u/L) TEST DATE: UNK	ALPHA HCV by PCR TEST DATE: UNK	ZEPTOMETRIX HCVHCV RT-PCR Copies/ML TEST DATE: UNK	VENDOR HBsAg TEST DATE: UNK	VENDOR VENDOR ANTI-HIV 1/2 TEST DATE: UNK	VENDOR VENDOR HIV p24Ag TEST DATE: UNK
10235-01	01/09/98	12	NEGATIVE	NOT DETECTED	NEGATIVE	NEGATIVE	NEGATIVE
10235-02	01/11/98	12	NEGATIVE	NOT DETECTED	NEGATIVE	NEGATIVE	NEGATIVE
10235-03	04/15/98	116	POSITIVE	1130000	NEGATIVE	NEGATIVE	NEGATIVE
10235-04	04/20/98	144	POSITIVE	1810000	NEGATIVE	NEGATIVE	NEGATIVE
10235-05	04/22/98	128	POSITIVE	3280000	NEGATIVE	NEGATIVE	NEGATIVE

PANEL MEMBER	VENDOR ANTI-HCV TEST DATE: UNK	ABBOTT ANTI-HCV TEST DATE: 8/13/1999	ORTHO ECi HBsAg TEST DATE: 11/16/2018	ORTHO ECi ANTI-HCV TEST DATE: 11/16/2018	ORTHO ECi ANTI-HIV 1/2 TEST DATE: 11/16/2018
10235-01	NEGATIVE	0.185	0.06	0.04	0.11
10235-02	NEGATIVE	0.222	0.05	0.04	0.10
10235-03	POSITIVE	5.778	0.05	20.1	0.09
10235-04	POSITIVE	7.074	0.05	21.1	0.10
10235-05	POSITIVE	7.037	0.05	21.1	0.10

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHCV10235 Rev03
Effective Date: 10/15/2021