

# ZeptoMetrix®

## HCV Seroconversion Panel Donor No. 62999 Part Number: HCV6225

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45 u/L)	ABBOTT MUREX ANTI-HCV 4.0	ABBOTT MUREX HCV Ag/Ab Combo	ABBOTT Corzyme ANTI-HBc	ABBOTT ANTI-HCV 2.0 Lab # 1	ABBOTT ANTI-HCV 2.0 Lab # 2	ABBOTT AxSYM ANTI-HCV	ABBOTT Architect ANTI-HCV
		TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 10/23/1997	TEST DATE: 11/6/1997	TEST DATE: 10/21/1997	TEST DATE: UNK
6225-01	10/25/96	10	0.10	0.314	2.50	0.2	0.17	0.47	0.08
6225-02	10/28/96	10	0.10	0.324	2.18	0.18	0.15	0.46	0.06
6225-03	11/01/96	6	0.10	0.324	2.48	0.18	0.14	0.50	0.07
6225-04	11/06/96	6	0.11	0.324	2.22	0.18	0.15	0.46	0.07
6225-05	11/08/96	8	0.11	0.334	2.53	0.15	0.15	0.42	0.06
6225-06	11/13/96	11	0.12	0.334	2.15	0.18	0.16	0.45	0.06
6225-07	11/19/96	8	0.11	0.331	2.10	0.15	0.15	0.46	0.07
6225-08	11/22/96	11	0.15	0.318	2.27	0.18	0.15	0.51	0.07
6225-09	11/26/96	0	0.11	0.311	2.64	0.19	0.15	0.47	0.07
6225-10	11/29/96	3	0.11	0.297	2.53	0.19	0.14	0.70	0.08
6225-11	12/03/96	4	0.11	0.301	2.44	0.19	0.16	0.65	0.07
6225-12	12/09/96	15	0.11	2.372	2.40	0.20	0.16	0.77	0.08
6225-13	12/11/96	3	0.11	8.105	2.33	0.21	0.17	0.65	0.07
6225-14	12/16/96	22	0.14	6.182	2.46	0.20	0.15	0.59	0.08
6225-15	12/20/96	11	0.13	6.301	2.19	0.19	0.15	0.53	0.07
6225-16	12/24/96	12	0.12	11.372	2.12	0.19	0.16	0.55	0.06
6225-17	01/06/97	288	0.16	1.791	2.00	0.25	0.17	0.97	0.18
6225-18	01/11/97	248	0.60	2.115	2.02	0.54	0.35	3.51	2.21
6225-19	01/13/97	248	1.17	1.655	2.17	0.60	0.47	5.75	3.01

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)

PIHCV6225 Rev03  
Effective Date: 06/15/2021

# ZeptoMetrix®

## HCV Seroconversion Panel Donor No. 62999 Part Number: HCV6225

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	ABBOTT PRISM ANTI-HCV TEST DATE: UNK	ORTHO ELISA ANTI-HCV 3.0 TEST DATE: UNK	ORTHO RIBA 2.0 TEST DATE: UNK	ORTHO RIBA 3.0 C100 TEST DATE: UNK	ORTHO RIBA 3.0 C33 TEST DATE: UNK	ORTHO RIBA 3.0 C22 TEST DATE: UNK	ORTHO RIBA 3.0 NS5 TEST DATE: UNK
6225-01	0.129	0.003	-	-	-	-	-
6225-02	0.178	0.002	-	-	-	-	-
6225-03	0.197	0.003	-	-	-	-	-
6225-04	0.225	0.003	-	-	-	-	-
6225-05	0.088	0.002	-	-	-	-	-
6225-06	0.097	0.003	-	-	-	-	-
6225-07	0.163	0.002	-	-	-	-	-
6225-08	0.121	0.003	-	-	-	-	-
6225-09	0.156	0.003	-	-	-	-	-
6225-10	0.127	0.003	-	-	-	-	-
6225-11	0.152	0.003	-	-	-	-	-
6225-12	0.164	0.002	-	-	-	-	-
6225-13	0.127	0.002	-	-	-	-	-
6225-14	0.124	0.003	-	-	-	-	-
6225-15	0.135	0.007	-	-	-	-	-
6225-16	0.104	0.003	-	-	-	-	-
6225-17	0.197	0.072	-	-	-	-	-
6225-18	2.044	1.734	-	-	1+	-	-
6225-19	2.630	2.110	-	-	1+	-	-

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)

# ZeptoMetrix®

## HCV Seroconversion Panel Donor No. 62999 Part Number: HCV6225

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	ORTHO RIBA 3.0 SOD TEST DATE: UNK	CHIRON bDNA (MEq/mL) TEST DATE: UNK	ROCHE HCV MONITOR (copies/mL) TEST DATE: UNK	PCR RNA Quantitative Result Lab # 3 TEST DATE: UNK	ORTHO ECi HBsAg TEST DATE: 11/16/2018	ORTHO ECi ANTI-HCV TEST DATE: 11/16/2018	ORTHO ECi ANTI-HIV ½ TEST DATE: 11/16/2018
6225-01	-	<0.2	<175	<100	0.05	0.02	0.13
6225-02	-	<0.2	<179	<100	0.06	0.02	0.13
6225-03	-	<0.2	<226	<100	0.06	0.02	0.12
6225-04	-	<0.2	<135	<100	0.06	0.02	0.13
6225-05	-	<0.2	<149	<100	0.06	0.02	0.12
6225-06	-	<0.2	<147	<100	0.06	0.02	0.13
6225-07	-	<0.2	<182	<100	0.06	0.01	0.13
6225-08	-	<0.2	<154	<100	0.06	0.02	0.12
6225-09	-	<0.2	<148	<100	0.06	0.02	0.13
6225-10	-	<0.2	<155	<100	0.06	0.02	0.13
6225-11	-	<0.2	670	<100	0.06	0.02	0.12
6225-12	-	27.02	390000	3000000	0.06	0.02	0.13
6225-13	-	98.84	1300000	4300000	0.05	0.02	0.14
6225-14	-	77.79	1300000	2900000	0.05	0.02	0.14
6225-15	-	54.71	810000	3100000	0.05	0.02	0.14
6225-16	-	>120	2400000	5000000	0.06	0.01	0.16
6225-17	-	9.373	510000	4200000	0.06	0.15	0.12
6225-18	-	4.971	460000	1700000	0.06	2.74	0.12
6225-19	-	3.242	230000	1600000	0.06	3.68	0.12

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)