



HCV Seroconversion Panel Donor No. 62886
Part Number: HCV6228

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

HCV Genotype: 1a

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45 u/L)	ABBOTT MUREX ANTI-HCV 4.0	ABBOTT MUREX HCV Ag/Ab Combo	ABBOTT Corzyme ANTI-HBc	ABBOTT ANTI-HCV 2.0 Lab # 1	ABBOTT ANTI-HCV 2.0 Lab # 2	ABBOTT Architect ANTI-HCV	ABBOTT PRISM ANTI-HCV
		TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 11/6/1997	TEST DATE: UNK	TEST DATE: UNK
6228-01	11/20/96	41	0.142	0.584	1.60	0.132	0.08	0.030	0.062
6228-02	11/22/96	26	0.127	0.416	1.42	0.134	0.11	0.040	0.081
6228-03	11/27/96	32	0.134	1.027/0.936	1.31	0.145	0.13	0.040	0.078
6228-04	11/29/96	36	0.128	0.580	1.46	0.120	0.12	0.030	0.066
6228-05	12/04/96	100	0.130	0.352	1.24	0.155	0.13	0.040	0.076
6228-06	12/06/96	248	0.110	0.297	1.50	0.128	0.11	0.030	0.080
6228-07	12/11/96	47	0.131	0.485	1.80	0.143	0.12	0.090	0.163
6228-08	12/14/96	196	0.124	0.608	1.72	0.128	0.14	0.100	0.137
6228-09	12/18/96	156	0.422	0.556	1.72	0.181	0.19	1.370	1.392
6228-10	12/21/96	53	0.985	1.092	1.59	0.252	0.21	4.520	4.582
6228-11	12/26/96	100	1.673	1.666	1.35	0.290	0.32	6.620	6.224
6228-12	12/28/96	96	1.679	1.529	1.44	0.370	0.31	7.120	6.665

PANEL MEMBER	ROCHE DIAGNOSTICS Cobas ANTI-HCV	ABBOTT AxSYM ANTI-HCV	ZEPTOMETRIX HCVRT PCR	ORTHO ANTI-HCV 3.0 (SAVe)	ORTHO ELISA ANTI-HCV 3.0	ORTHO ECi ANTI-HIV	ORTHO RIBA 2.0	ORTHO RIBA 3.0 C100
	TEST DATE: 11/7/1997	TEST DATE: 10/21/1997	TEST DATE: UNK	TEST DATE: 10/14/1997	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK
6228-01	0.09	0.52	150000	0.01	0.000	0.03	-	-
6228-02	0.09	0.47	46300	0.01	0.000	0.03	-	-
6228-03	0.1	0.46	138000	0.01	0.002	0.03	-	-
6228-04	0.1	0.50	49600	0.01	0.000	0.03	-	-
6228-05	0.09	0.47	35400	0.01	0.000	0.03	-	-
6228-06	0.1	0.47	1840	0.01	0.010	0.03	-	-
6228-07	0.1	0.54	61000	0.01	0.007	0.05	-	-
6228-08	0.09	0.51	169000	0.01	0.007	0.08	-	-
6228-09	0.06	0.52	407	0.28	0.642	1.33	-	-
6228-10	0.1	0.63	816	1.10	2.166	4.03	-	-
6228-11	0.15	0.62	38800	1.31	2.842	4.50	-	-
6228-12	0.17	0.62	3160	1.48	3.192	4.84	-	-

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHCV6228 Rev04
Effective Date: 04/24/2024



HCV Seroconversion Panel Donor No. 62886
Part Number: HCV6228

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	ORTHO RIBA 3.0 C33 TEST DATE: UNK	ORTHO RIBA 3.0 C22 TEST DATE: UNK	ORTHO RIBA 3.0 NS5 TEST DATE: UNK	ORTHO RIBA 3.0 SOD TEST DATE: UNK	ORTHO RIBA 3.0 TEST DATE: 10/29/1997	CHIRON bDNA (MEq/mL) TEST DATE: 9/23/1997	ROCHE HCV MONITOR (copies/mL) TEST DATE: UNK	PCR RNA Quantitative Result Lab # 3 TEST DATE: 10/25/1997
6228-01	-	-	-	-	-	1.486	270000	670000
6228-02	-	-	-	-	-	0.986	180000	260000
6228-03	-	-	-	-	-	2.480	320000	1000000
6228-04	-	-	-	-	-	0.916	400000	445000
6228-05	-	-	-	-	-	<0.2	99000	240000
6228-06	-	-	-	-	-	<0.2	5300	15500
6228-07	-	-	-	-	-	0.898	160000	1000000
6228-08	-	-	-	-	-	1.625	140000	1300000
6228-09	-	-	-	-	-	<0.2	340	500
6228-10	1+	-	-	-	INDETERMINATE	<0.2	1600	1000
6228-11	2+	-	-	-	INDETERMINATE	0.396	78000	440000
6228-12	2+	-	-	-	INDETERMINATE	<0.2	3000	7500

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHCV6228 Rev04
Effective Date: 04/24/2024