



**HCV Seroconversion Panel Donor No. 63792**  
**Part Number: HCV9042**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
 Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45 u/L)  TEST DATE: UNK	ABBOTT MUREX ANTI-HCV 4 0  TEST DATE: UNK	ABBOTT MUREX HCV Ag/Ab Combo  TEST DATE: UNK	ABBOTT ANTI-HCV Lab # 1  TEST DATE: 5/20/1998	ABBOTT ANTI-HCV Lab # 2  TEST DATE: 6/15/1998	ABBOTT ANTI-HCV 3.0  TEST DATE: 7/8/1998	ORTHO ANTI-HCV 3.0  TEST DATE: UNK
9042-01	04/18/97	11	0.167	0.341	0.149	0.251	0.17	0.028
9042-02	04/23/97	19	0.268	0.608	0.823	0.547	0.43	0.126
9042-03	04/26/97	19	1.348	0.942	3.333	3.549	2.55	1.980
9042-04	05/01/97	33	1.160	0.386	3.377	3.622	2.44	2.044
9042-05	05/07/97	22	1.158	0.328	3.528	3.444	1.91	1.959
9042-06	05/11/97	25	0.798	0.334	3.420	2.872	1.81	1.547

PANEL MEMBER	ABBOTT PRISM HCV  TEST DATE: 8/5/1998	ABBOTT IMx HCV 3.0 Reagent Pack  TEST DATE: 8/4/1998	ABBOTT AxSYM ANTI-HCV 3.0 Reagent Pack  TEST DATE: 7/6/1998	ABBOTT Architect ANTI-HCV  TEST DATE: UNK	ABBOTT PRISM ANTI-HCV  TEST DATE: UNK	ROCHE/ BOEHRINGER MANNHEIM Cobas Core ANTI-HCV EIA  TEST DATE: 7/7/1998	ORTHO Enhanced SAVe HCV 3.0 Lab # 4  TEST DATE: 6/29/1998	ORTHO Enhanced SAVe HCV 3.0 Lab # 7  TEST DATE: 6/23/1998
9042-01	0.07	0.23	0.18	0.140	0.192	0.1	0.04	0.029
9042-02	0.40	0.26	0.35	0.890	0.543	0.1	0.40	0.271
9042-03	1.71	0.65	1.42	4.240	2.189	0.2	4.19	2.829
9042-04	1.44	0.69	1.38	4.550	2.352	0.2	4.88	3.727
9042-05	1.63	0.73	0.99	4.510	2.043	0.2	4.71	3.466
9042-06	1.53	0.53	0.94	4.070	2.015	0.1	3.74	2.118

PANEL MEMBER	SANOFI Monolisa ANTI- HCV PLUS  TEST DATE: 6/30/1998	SANOFI Monolisa ANTI- HCV PLUS Lab # 7  TEST DATE: 6/23/1998	ORTHO Chiron Riba HCV 2.0 5-1-1(NS4)  TEST DATE: 7/7/1998	ORTHO Chiron Riba HCV 2.0 C100(NS4)  TEST DATE: 7/7/1998	ORTHO Chiron Riba HCV 2.0 C33(NS3)  TEST DATE: 7/7/1998	ORTHO Chiron Riba HCV 2.0 C22(Core)  TEST DATE: 7/7/1998	ORTHO Chiron Riba HCV 2.0 SOD)  TEST DATE: 7/7/1998	ORTHO Chiron Riba HCV 3.0 c100(p)  TEST DATE: 7/7/1998
9042-01	0.12	0.095	NEGATIVE	NEGATIVE	-	-	-	-
9042-02	0.25	0.082	NEGATIVE	NEGATIVE	1+	-	-	-
9042-03	1.72	0.251	NEGATIVE	NEGATIVE	4+	-	-	-
9042-04	1.68	0.257	NEGATIVE	NEGATIVE	4+	-	-	-
9042-05	1.89	0.164	NEGATIVE	NEGATIVE	4+	-	-	-
9042-06	1.34	0.130	NEGATIVE	NEGATIVE	4+	-	-	-

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
 This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
 For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)

PIHCV9042 Rev03  
 Effective Date: 10/08/2021

# ZeptoMetrix®

## HCV Seroconversion Panel Donor No. 63792 Part Number: HCV9042

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	ORTHO Chiron Riba HCV 3.0 c33c  TEST DATE: 7/7/1998	ORTHO Chiron Riba HCV 3.0 c22(p)  TEST DATE: 7/7/1998	ORTHO Chiron Riba HCV 3.0 NS5  TEST DATE: 7/7/1998	ORTHO Chiron Riba HCV 3.0 Interpretation  TEST DATE: 7/7/1998	CHIRON HCV RNA bDNA 2.0 (MEq/mL*)  TEST DATE: 7/16/1998	ORTHO ECi HBsAg  TEST DATE: 11/16/2018	ORTHO ECi ANTI-HCV  TEST DATE: 11/16/2018	ORTHO ECi ANTI-HIV ½  TEST DATE: 11/16/2018
9042-01	-	-	+/-	-	<0.2	0.05	0.08	0.10
9042-02	1+	-	+/-	INDETERMINATE	<0.2	0.05	0.53	0.11
9042-03	4+	-	+/-	INDETERMINATE	<0.2	0.05	5.03	0.10
9042-04	4+	-	+/-	INDETERMINATE	<0.2	0.05	5.28	0.12
9042-05	4+	-	+/-	INDETERMINATE	<0.2	0.05	4.87	0.13
9042-06	4+	-	+/-	INDETERMINATE	<0.2	0.06	4.07	0.12

### Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)

PIHCV9042 Rev03  
Effective Date: 10/08/2021