



**HCV Seroconversion Panel Donor No. 63869**  
**Part Number: HCV9043**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
 Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ALT	ABBOTT MUREX ANTI-HCV 4 0	ABBOTT MUREX HCV Ag/Ab Combo	ABBOTT ANTI-HCV Lab # 1	ABBOTT ANTI-HCV Lab # 2	ABBOTT ANTI-HCV 3.0	ORTHO ANTI-HCV 3.0
		Normal Range (10-45 u/L)	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 5/20/1998	TEST DATE: UNK	TEST DATE: 7/8/1998
9043-01	04/28/97	6	0.161	0.365	1.059	1.178	1.64	0.692
9043-02	05/05/97	7	0.229	0.365	1.181	1.149	1.97	0.723
9043-03	05/08/97	6	0.172	0.341	1.094	1.064	1.78	0.614
9043-04	05/12/97	5	0.166	0.328	1.163	1.073	1.55	0.753
9043-05	05/15/97	N/A	0.148	0.345	0.955	0.977	1.72	0.573
9043-06	05/19/97	4	0.131	0.328	0.920	0.911	1.30	0.543

PANEL MEMBER	ABBOTT PRISM ANTI-HCV	ABBOTT IMx HCV HCV 3.0 Reagent Pack	ABBOTT AxSYM ANTI-HCV 3.0 Reagent Pack	ABBOTT Architect ANTI-HCV	ABBOTT PRISM ANTI-HCV	ROCHE/BOEHRINGER MANNHEIM Cobas Core ANTI-HCV EIA	ORTHO VITROS ECi ANTI-HCV	ORTHO Enhanced SAVE HCV 3.0 Lab # 4
	TEST DATE: 8/5/1998	TEST DATE: 8/4/1998	TEST DATE: 7/8/1998	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 7/7/1998	TEST DATE: UNK	TEST DATE: 6/29/1998
9043-01	0.59	0.53	0.25	0.880	0.692	1.6	3.03	0.66
9043-02	0.44	0.55	0.24	1.020	0.584	2.0	3.36	1.36
9043-03	0.46	0.54	0.26	0.840	0.624	1.5	3.01	1.14
9043-04	0.50	0.49	0.28	0.860	0.487	1.8	2.90	0.74
9043-05	0.46	0.51	0.20	0.640	0.610	2.4	2.75	0.99
9043-06	0.43	0.45	0.29	0.710	0.498	1.4	1.91	0.89

PANEL MEMBER	ORTHO Enhanced SAVE HCV 3.0 Lab # 7	SANOFI Monalisa ANTI- HCV PLUS	SANOFI Monalisa ANTI- HCV PLUS Lab # 7	ORTHO Chiron Riba HCV 2.0 5-1-1 (NS4)	ORTHO Chiron Riba HCV 2.0 C100 (NS4)	ORTHO Chiron Riba HCV 2.0 C33 (NS3)	ORTHO Chiron Riba HCV 2.0 C22 (Core)
	TEST DATE: 6/23/1998	TEST DATE: 6/30/1998	TEST DATE: 6/23/1998	TEST DATE: 7/7/1998	TEST DATE: 7/7/1998	TEST DATE: 7/7/1998	TEST DATE: 7/7/1998
9043-01	0.572	0.24	0.070	-	-	-	+/-
9043-02	0.339	0.46	0.045	-	-	-	+/-
9043-03	0.344	0.34	0.087	-	-	-	+/-
9043-04	0.369	0.31	0.053	-	-	-	+/-
9043-05	0.341	0.39	0.055	-	-	-	+/-
9043-06	0.487	0.44	0.053	-	-	-	+/-

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

**This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.**

ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
 For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [custserv@zeptometrix.com](mailto:custserv@zeptometrix.com)



**HCV Seroconversion Panel Donor No. 63869**  
**Part Number: HCV9043**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
 Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	ORTHO Chiron Riba HCV 2.0 SOD)	ORTHO Chiron Riba HCV 3.0 c100(p)	ORTHO Chiron Riba HCV 3.0 c33c	ORTHO Chiron Riba HCV 3.0 c22(p)	ORTHO Chiron Riba HCV 3.0 NS5	ORTHO Chiron Riba HCV 3.0 Interpretation	CHIRON HCV RNA bDNA 2.0 (MEq/mL*)
	TEST DATE: 7/7/1998	TEST DATE: 7/7/1998	TEST DATE: 7/7/1998	TEST DATE: 7/7/1998	TEST DATE: 7/7/1998	TEST DATE: 7/7/1998	TEST DATE: 7/16/1998
9043-01	-	-	-	1+	-	INDETERMINATE	<0.2
9043-02	-	-	-	1+	-	INDETERMINATE	<0.2
9043-03	-	-	-	1+	-	INDETERMINATE	<0.2
9043-04	-	-	-	1+	-	INDETERMINATE	<0.2
9043-05	-	-	-	1+	-	INDETERMINATE	<0.2
9043-06	-	-	-	1+	-	INDETERMINATE	<0.2

www.h-h-c.com

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

**This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.**

ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
 For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [custserv@zeptometrix.com](mailto:custserv@zeptometrix.com)