

ZeptoMetrix®

HCV Seroconversion Panel Donor No. 63911 Part Number: HCV9044

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

| PANEL MEMBER | DATE OF DRAW | ALT Normal Range (10-45 u/L) | ABBOTT MUREX ANTI-HCV 4 0 | ABBOTT MUREX HCV Ag/Ab Combo | ABBOTT ANTI-HCV Lab # 1 | ABBOTT ANTI-HCV Lab # 2 | ABBOTT ANTI-HCV 3.0 | ORTHO ANTI-HCV 3.0 |
|--------------|--------------|------------------------------------|------------------------------------|---------------------------------------|-------------------------------|-------------------------------|---------------------------|--------------------------|
| | | TEST DATE: UNK | TEST DATE: UNK | TEST DATE: UNK | TEST DATE: 5/20/1998 | TEST DATE: 6/15/1998 | TEST DATE: 7/8/1998 | TEST DATE: UNK |
| 9044-01 | 04/14/97 | 14 | 0.106 | 8.846 | 0.045 | 0.219 | 0.08 | 0.005 |
| 9044-02 | 04/18/97 | 13 | 0.106 | 6.184 | 0.048 | 0.226 | 0.09 | 0.003 |
| 9044-03 | 05/01/97 | 28 | 0.166 | 7.693 | 0.039 | 0.180 | 0.10 | 0.003 |
| 9044-04 | 05/05/97 | 108 | 0.611 | 6.655 | 0.117 | 0.319 | 0.23 | 0.124 |
| 9044-05 | 05/09/97 | 124 | 2.548 | 7.133 | 0.693 | 1.586 | 1.15 | 1.364 |
| 9044-06 | 05/13/97 | 172 | 3.965 | 7.676 | 1.169 | 2.100 | 1.72 | 1.864 |

| PANEL MEMBER | ABBOTT PRISM HCV | ABBOTT IMx HCVHCV 3.0 Reagent Pack | ABBOTT AxSYM ANTI-HCV 3.0 Reagent Pack | ABBOTT Architect ANTI-HCV | ABBOTT PRISM ANTI-HCV | ROCHE/ BOEHRINGER MANNHEIM Cobas Core ANTI-HCV EIA | ORTHO VITROS ECI ANTI-HCV |
|--------------|------------------------|---|--|---------------------------------|-----------------------------|--|------------------------------------|
| | TEST DATE: 8/5/1998 | TEST DATE: 8/4/1998 | TEST DATE: 7/6/1998 | TEST DATE: UNK | TEST DATE: UNK | TEST DATE: 7/7/1998 | TEST DATE: UNK |
| 9044-01 | 0.06 | 0.17 | 0.13 | 0.050 | 0.090 | 0.1 | 0.005 |
| 9044-02 | 0.08 | 0.17 | 0.10 | 0.080 | 0.090 | 0.1 | 0.003 |
| 9044-03 | 0.07 | 0.19 | 0.16 | 0.070 | 0.120 | 0.1 | 0.003 |
| 9044-04 | 0.93 | 0.46 | 0.71 | 1.250 | 0.830 | 0.6 | 0.124 |
| 9044-05 | 3.42 | 1.73 | 4.40 | 5.430 | 3.900 | 3.3 | 1.364 |
| 9044-06 | 4.07 | 3.66 | 11.95 | 7.230 | 4.780 | 5.8 | 1.864 |

| PANEL MEMBER | ORTHO Enhanced SAve HCV 3.0 Lab # 4 | ORTHO Enhanced SAve HCV 3.0 Lab # 7 | SANOFI Monolisa ANTI- HCV PLUS | SANOFI Monolisa ANTI- HCV PLUS Lab # 7 | ORTHO Chiron Riba HCV 2.0 5-1-1(NS4) | ORTHO Chiron Riba HCV 2.0 C100(NS4) | ORTHO Chiron Riba HCV 2.0 C33(NS3) |
|--------------|--|--|---|---|---|--|---|
| | TEST DATE: 6/29/1998 | TEST DATE: 6/23/1998 | TEST DATE: 6/30/1998 | TEST DATE: 6/23/1998 | TEST DATE: 7/7/1998 | TEST DATE: 7/7/1998 | TEST DATE: 7/7/1998 |
| 9044-01 | 0.00 | 0.009 | 0.38 | 0.097 | - | - | - |
| 9044-02 | 0.01 | 0.024 | 0.42 | 0.059 | - | - | - |
| 9044-03 | 0.00 | 0.021 | 0.43 | 0.073 | - | - | - |
| 9044-04 | 0.69 | 0.320 | 1.14 | 0.260 | - | - | +/- |
| 9044-05 | 3.70 | 1.860 | 3.05 | 1.358 | - | - | 1+ |
| 9044-06 | 4.51 | 3.160 | 3.60 | 2.258 | - | - | 2+ |

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHCV9044 Rev03
Effective Date: 06/15/2021

ZeptoMetrix®

HCV Seroconversion Panel Donor No. 63911 Part Number: HCV9044

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

| PANEL MEMBER | ORTHO Chiron Riba HCV 2.0 C22(Core) TEST DATE: 7/7/1998 | ORTHO Chiron Riba HCV 2.0 (SOD) TEST DATE: 7/7/1998 | ORTHO Chiron Riba HCV 3.0 c100(p) TEST DATE: 7/7/1998 | ORTHO Chiron Riba HCV 3.0 c33c TEST DATE: 7/7/1998 | ORTHO Chiron Riba HCV 3.0 c22(p) TEST DATE: 7/7/1998 | ORTHO Chiron Riba HCV 3.0 NS5 TEST DATE: 7/7/1998/ | ORTHO Chiron Riba HCV 3.0 Interpretation TEST DATE: 7/7/1998 | CHIRON HCV RNA bDNA 2.0 (MEq/mL*) TEST DATE: 7/16/1998 |
|--------------|--|--|--|---|---|---|---|---|
| 9044-01 | - | - | - | - | - | - | - | 105.9 |
| 9044-02 | - | - | - | - | - | - | - | 52.71 |
| 9044-03 | - | - | - | - | - | - | - | 77.18 |
| 9044-04 | - | - | - | +/- | - | - | - | 71.52 |
| 9044-05 | - | - | - | 2+ | - | +/- | INDETERMINATE | 72.82 |
| 9044-06 | - | - | +/- | 3+ | - | +/- | INDETERMINATE | 48.83 |

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHCV9044 Rev03
Effective Date: 06/15/2021