

ZeptoMetrix®

HCV Seroconversion Panel Donor No. 64437 Part Number: HCV9048

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45 u/L) TEST DATE: UNK	ABBOTT MUREX ANTI-HCV 4.0 TEST DATE: UNK	ABBOTT MUREX HCV Ag/Ab Combo TEST DATE: UNK	ABBOTT ANTI-HCV Lab # 1 TEST DATE: 5/20/1998	ABBOTT ANTI-HCV Lab # 2 TEST DATE: 6/15/1998	ABBOTT ANTI-HCV 3.0 TEST DATE: 7/8/1998	ORTHO ANTI-HCV 3.0 TEST DATE: UNK	ABBOTT PRISM HCV TEST DATE: 8/5/1998
9048-01	07/14/97	15	0.716	0.614	0.866	0.811	0.96	0.061	0.79
9048-02	07/16/97	11	0.582	0.529	0.910	0.847	0.91	0.052	0.78
9048-03	07/21/97	16	0.504	0.567	0.952	0.777	0.91	0.123	0.95
9048-04	07/24/97	11	0.605	0.567	0.910	0.830	1.11	0.051	0.66
9048-05	07/29/97	21	0.724	0.635	1.125	0.920	1.47	0.080	0.87
9048-06	07/31/97	14	0.594	0.679	1.364	1.055	1.04	0.154	1.07
9048-07	08/05/97	5	0.685	0.635	1.125	1.011	0.93	0.229	0.91
9048-08	08/07/97	11	0.649	0.631	1.255	0.880	0.56	0.137	0.85

PANEL MEMBER	ABBOTT IMx HCV 3.0 Reagent Pack TEST DATE: 8/4/1998	ABBOTT AxSYM ANTI-HCV 3.0 Reagent Pack TEST DATE: 7/6/1998	ABBOTT Architect ANTI-HCV TEST DATE: UNK	ABBOTT PRISM ANTI-HCV TEST DATE: UNK	ROCHE/BOEHRINGER MANNHEIM Cobas Core ANTI-HCV EIA TEST DATE: 7/7/1998	ORTHO Eci ANTI-HCV TEST DATE: UNK	ORTHO Enhanced SAVE HCV 3.0 Lab # 4 TEST DATE: 7/15/1998	ORTHO Enhanced SAVE HCV 3.0 Lab # 7 TEST DATE: 6/25/1998	SANOFI Monolisa ANTI-HCV PLUS TEST DATE: 6/30/1998
9048-01	0.85	0.82	1.110	0.819	3.5	1.16	1.22	0.490	1.42
9048-02	0.69	0.71	0.930	0.883	2.9	1.14	1.29	0.560	1.68
9048-03	0.63	0.77	0.860	0.809	2.9	1.03	1.11	0.334	1.49
9048-04	0.77	0.68	1.000	0.931	2.8	1.05	1.36	0.451	1.61
9048-05	0.72	0.78	1.660	1.192	3.9	2.05	1.84	0.757	2.04
9048-06	0.85	0.89	1.350	1.294	4.0	1.49	2.29	0.381	2.62
9048-07	0.76	0.82	1.340	1.231	3.8	1.39	1.49	0.674	1.99
9048-08	0.79	0.71	1.200	1.204	3.7	1.07	1.18	0.290	1.59

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHCV9048 Rev03
Effective Date: 10/11/2021



HCV Seroconversion Panel Donor No. 64437
Part Number: HCV9048

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
 Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	SANOFI Monolisa ANTI- HCV PLUS Lab # 7	ORTHO Chiron Riba HCV 2.0 5-1-1(NS4)	ORTHO Chiron Riba HCV 2.0 C100(NS4)	ORTHO Chiron Riba HCV 2.0 C33(NS3)	ORTHO Chiron Riba HCV 2.0 C22(Core)	ORTHO Chiron Riba HCV 2.0 SOD	ORTHO Chiron RIBA HCV 2.0 Interpretation	ORTHO Chiron Riba HCV 3.0 c100(p)
	TEST DATE: 6/25/1998	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 7/22/1998
9048-01	0.725	-	+/-	+/-	-	-	-	+/-
9048-02	0.824	-	+/-	+/-	-	-	-	+/-
9048-03	0.574	-	+/-	+/-	-	-	-	+/-
9048-04	0.717	-	+/-	+/-	-	-	-	+/-
9048-05	1.132	-	+/-	1+	-	+/-	INDETERMINATE	+/-
9048-06	1.255	-	+/-	1+	-	+/-	INDETERMINATE	+/-
9048-07	1.006	-	+/-	1+	-	+/-	INDETERMINATE	+/-
9048-08	1.009	-	+/-	1+	-	+/-	INDETERMINATE	+/-

PANEL MEMBER	ORTHO Chiron Riba HCV 3.0 c33c	ORTHO Chiron Riba HCV 3.0 c22(p)	ORTHO Chiron Riba HCV 3.0 NS5	ORTHO Chiron Riba HCV 3.0 Interpretation	CHIRON HCV RNA bDNA 2.0 (MEq/mL*)	ORTHO ECi HBsAg	ORTHO ECi ANTI-HCV	ORTHO ECi ANTI-HIV ½
	TEST DATE: 7/22/1998	TEST DATE: 7/22/1998	TEST DATE: 7/22/1998	TEST DATE: 7/22/1998	TEST DATE: 7/16/1998	TEST DATE: 11/16/2018	TEST DATE: 11/16/2018	TEST DATE: 11/16/2018
9048-01	+/-	-	-	-	<0.2	0.05	0.61	0.13
9048-02	1+	-	-	INDETERMINATE	<0.2	0.05	0.78	0.12
9048-03	+/-	-	-	-	<0.2	0.05	0.72	0.11
9048-04	1+	-	-	INDETERMINATE	<0.2	0.05	0.81	0.10
9048-05	2+	-	+/-	INDETERMINATE	<0.2	0.06	1.53	0.10
9048-06	2+	-	+/-	INDETERMINATE	<0.2	0.05	1.39	0.11
9048-07	3+	-	+/-	INDETERMINATE	<0.2	0.05	1.21	0.11
9048-08	1+	-	+/-	INDETERMINATE	<0.2	0.05	0.88	0.12

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
 This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
 For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHCV9048 Rev03
 Effective Date: 10/11/2021