

ZeptoMetrix®

HCV Seroconversion Panel Donor No. 66626 Part Number: HCV9054

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

HCV Genotype: 3a

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45 u/L) TEST DATE: UNK	ABBOTT MUREX ANTI-HCV 4 0 TEST DATE: UNK	ABBOTT MUREX HCV Ag/Ab Combo TEST DATE: UNK	ABBOTT ANTI-HCV TEST DATE: 6/4/1999	ALPHA HCV by PCR (NGI) TEST DATE: UNK	CHIRON bDNA 2.0 (MEq/mL) TEST DATE: 10/27/1999
9054-01	12/23/97	7	0.116	0.334	0.183	NEGATIVE	<0.2
9054-02	01/09/98	7	0.136	0.341	0.209	NEGATIVE	<0.2
9054-03	01/11/98	4	0.110	0.334	0.209	NEGATIVE	<0.2
9054-04	01/28/98	13	0.118	0.341	0.183	NEGATIVE	<0.2
9054-05	01/30/98	14	0.112	0.314	0.157	NEGATIVE	<0.2
9054-06	02/07/98	14	0.124	0.297	0.157	NEGATIVE	<0.2
9054-07	02/13/98	17	0.113	0.300	0.183	NEGATIVE	<0.2
9054-08	03/07/98	40	0.567	2.917	0.235	POSITIVE	86.79
9054-09	03/10/98	116	4.151	10.883	0.705	POSITIVE	86.75
9054-10	03/15/98	216	5.410	11.924	3.290	POSITIVE	59.14

PANEL MEMBER	ABBOTT MUREX ANTI-HCV 4.0 TEST DATE: 6/16/1999	ABBOTT PRISM HCV TEST DATE: 6/15/1999	ABBOTT IMx HCV 3.0 Reagent Pack TEST DATE: 7/21/1999	ABBOTT AxSYM ANTI-HCV 3.0 Reagent Pack TEST DATE: 6/28/1999	ABBOTT Architect ANTI-HCV TEST DATE: UNK	ABBOTT PRISM ANTI-HCV TEST DATE: UNK	ORTHO VITROS ECi ANTI-HCV TEST DATE: UNK
9054-01	0.19	0.06	0.12	0.13	0.120	0.070	0.04
9054-02	0.13	0.07	0.09	0.15	0.080	0.060	0.05
9054-03	0.19	0.06	0.11	0.07	0.070	0.070	0.04
9054-04	0.15	0.06	0.06	0.15	0.060	0.070	0.04
9054-05	0.24	0.06	0.09	0.11	0.090	0.090	0.04
9054-06	0.29	0.06	0.10	0.09	0.080	0.070	0.04
9054-07	0.20	0.06	0.12	0.12	0.060	0.080	0.04
9054-08	0.66	0.09	0.11	0.17	0.240	0.090	0.06
9054-09	MAX ABS	0.25	0.22	0.36	1.470	0.350	1.19
9054-10	3.00	1.41	1.90	2.10	3.740	1.360	8.50

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHCV9054 Rev05
Effective Date: 10/11/2021

ZeptoMetrix®

HCV Seroconversion Panel Donor No. 66626 Part Number: HCV9054

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	ORTHO Enhanced SAve ANTI- HCV 3.0 TEST DATE: 6/9/1999	SANOFI Monolisa ANTI- HCV PLUS TEST DATE: 7/29/1999	ORTHO Chiron Riba HCV 3.0 c100(p)/5-1-1 TEST DATE: 2/3/2000	ORTHO Chiron Riba HCV 3.0 c33c TEST DATE: 2/3/2000	ORTHO Chiron Riba HCV 3.0 c22(p) TEST DATE: 2/3/2000	ORTHO Chiron Riba HCV 3.0 NS5 TEST DATE: 2/3/2000	ORTHO Chiron Riba HCV 3.0 Interpretation TEST DATE: 2/3/2000
9054-01	0.00	0.04	-	-	-	-	-
9054-02	0.00	0.02	-	-	-	-	-
9054-03	0.00	0.03	-	-	-	-	-
9054-04	0.00	0.02	-	-	-	-	-
9054-05	0.00	0.04	-	-	-	-	-
9054-06	0.00	0.04	-	-	-	-	-
9054-07	0.00	0.04	-	-	-	-	-
9054-08	0.00	0.10	-	-	+/-	-	-
9054-09	0.14	0.74	+/-	+/-	1+	-	INDETERMINATE
9054-10	3.16	4.93	-	1+	2+	-	+

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHCV9054 Rev05
Effective Date: 10/11/2021