

ZeptoMetrix®

HCV Seroconversion Panel Donor No. 66647 Part Number: HCV9058

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

HCV Genotype: 1a

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45 u/L) TEST DATE: 3/26/1999	ABBOTT MUREX ANTI-HCV 4.0 TEST DATE: UNK	ABBOTT MUREX HCV Ag/Ab Combo TEST DATE: UNK	ALPHA HCV by PCR (NGI) TEST DATE: UNK	CHIRON bDNA TEST DATE: 10/27/1999	ABBOTT ANTI-HCV TEST DATE: 6/4/1999	ABBOTT Architect ANTI-HCV TEST DATE: UNK	ABBOTT PRISM ANTI-HCV TEST DATE: UNK
9058-01	03/06/98	19	0.303	10.972	POSITIVE	90.58	0.679	0.270	0.243
9058-02	03/09/98	18	0.313	11.524	POSITIVE	85.34	0.653	0.290	0.344
9058-03	03/13/98	55	0.457	10.583	POSITIVE	37.64	0.653	0.720	0.616
9058-04	03/16/98	75	1.185	10.852	POSITIVE	48.33	0.862	1.970	1.995
9058-05	03/20/98	156	2.900	12.293	POSITIVE	29.46	1.619	4.440	4.126

PANEL MEMBER	ORTHO Enhanced SAVE ANTI-HCV 3.0 TEST DATE: 2/29/2000	ORTHO Chiron RIBA HCV 3.0 c100(p)/5-1-1 TEST DATE: 2/3/2000	ORTHO Chiron RIBA HCV 3.0 c33c TEST DATE: 2/3/2000	ORTHO Chiron RIBA HCV 3.0 c22(p) TEST DATE: 2/3/2000	ORTHO Chiron RIBA HCV 3.0 NS5 TEST DATE: 2/3/2000	ORTHO Chiron RIBA HCV 2.0 Interpretation TEST DATE: 1/17/2000	ORTHO ECi HBsAg TEST DATE: 11/16/2018	ORTHO ECi ANTI-HCV TEST DATE: 11/16/2018	ORTHO ECi ANTI-HIV ½ TEST DATE: 11/16/2018
9058-01	0.073	-	-	+/-	-	-	0.07	0.25	0.17
9058-02	0.078	-	-	+/-	-	-	0.05	0.20	0.15
9058-03	0.147	-	+/-	+	-	-	0.05	0.35	0.15
9058-04	0.660	-	1+	1+	-	+	0.05	1.63	0.15
9058-05	3.163	-	1+	2+	-	+	0.06	5.21	0.13

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHCV9058 Rev02
Effective Date: 10/11/2021