

ZeptoMetrix®

HIV Seroconversion Panel Donor No. 63331 Part Number: HIV6248

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	CHIRON HIV-1 bDNA TEST DATE: 11/11/1997	ROCHE COBAS AmpliCor HIV-1 Monitor TEST DATE: UNK	ABBOTT HIV-1 p24 Ag NT % TEST DATE: 11/6/1997	COULTER HIV-1 p24 Ag TEST DATE: 10/11/1997	COULTER HIV-1 p24 Ag NT% TEST DATE: UNK	ABBOTT AxSYM HIV AB ½ EIA TEST DATE: 10/15/1997	ABBOTT HIV AB ½ EIA rDNA TEST DATE: UNK	ABBOTT MUREX HIV Ab ½ + O 3 RD Generation TEST DATE: UNK	ABBOTT PRISM HIV Ag/Ab COMBO 4 th Generation TEST DATE: UNK
6248-01	02/12/97	<100	<400	NEGATIVE	0.275	0.072	0.46	0.195	0.384	0.08
6248-02	02/16/97	<500	<400	NEGATIVE	0.245	0.101	0.44	0.136	0.410	0.11
6248-03	02/19/97	<500	<400	NEGATIVE	0.245	0.072	0.38	0.059	0.466	0.21
6248-04	02/23/97	<500	<400	NEGATIVE	0.245	0.072	0.38	0.093	0.370	0.11
6248-05	02/26/97	561	2300	NEGATIVE	0.265	0.072	0.39	0.085	0.354	0.10
6248-06	03/02/97	74220	199000	1.17	2.696	2.899	0.34	0.102	0.325	2.01
6248-07	03/09/97	>800000	87700000	>86.3	38.255	36.232	1.35	1.508	5.230	293.16

PANEL MEMBER	ABBOTT MUREX HIV-1 Ab/Ag COMBO 4 th Generation TEST DATE: UNK	ABBOTT HIV Ag/Ab COMBO ARCHITECT 4 th Generation TEST DATE: UNK	BOEHRINGER MANNHEIM ENZYMNTEST ANTI-HIV ½ PLUS SUBTYPE O TEST DATE: 10/8/1997	BIOMERIEUX ORGANON VIRONOSTIKA HIV UNIFORM II PLUS O TEST DATE: 10/22/1997	BIOTEST ANTI-HIV ½ TEST DATE: 10/23/1997	DADE BEHRING ENZYGNOST ANTI-HIV ½ PLUS TEST DATE: 10/28/1997	GENETICS SYSTEMS rLAV EIA TEST DATE: 11/20/1997	ORTHO ANTI-HIV ½ ENHANCED ELISA TEST DATE: 10/28/1997
6248-01	0.230	0.12	0.36	0.27	0.16	0.19	0.097	0.02
6248-02	0.260	0.15	0.36	0.30	0.15	0.16	0.224	0.02
6248-03	0.235	0.12	0.37	0.29	0.14	0.20	0.173	0.02
6248-04	0.235	0.12	0.37	0.30	0.17	0.29	0.134	0.02
6248-05	0.240	0.11	0.36	0.39	0.18	0.28	0.108	0.02
6248-06	1.670	1.855	0.64	0.41	0.17	0.16	0.134	0.02
6248-07	17.990	464.579	1.14	1.08	0.34	0.69	0.097	0.03

PANEL MEMBER	CAMBRIDGE BIOTECH HIV-1 WESTERN BLOT TEST DATE: UNK	WALDHEIM DIAGNOSTICA ANTI-HIV-1 TEST DATE: UNK	SANOBI GENSCREEN ANTI-HIV ½ TEST DATE: 10/30/1997	BIO-RAD BioPlex 2200 HIV Ag-Ab 5 th Generation TEST DATE: UNK	BIO-RAD BioPlex 2200 HIV-1 Ag 5 th Generation TEST DATE: UNK	BIO-RAD BioPlex 2200 HIV-1 Ab 5 th Generation TEST DATE: UNK	ORTHO VITROS EC1 ANTI-HIV ½ TEST DATE: 8/30/2018
6248-01	NEGATIVE	NEGATIVE	0.18	0.69	0.06	0.69	0.94
6248-02	NEGATIVE	NEGATIVE	0.14	0.10	0.08	0.06	0.10
6248-03	NEGATIVE	NEGATIVE	0.19	0.12	0.12	0.04	0.11
6248-04	NEGATIVE	NEGATIVE	0.19	0.15	0.15	0.06	0.10
6248-05	NEGATIVE	NEGATIVE	0.16	0.13	0.13	0.03	0.09
6248-06	NEGATIVE	NEGATIVE	0.19	7.82	7.82	0.06	0.09
6248-07	NEGATIVE	NEGATIVE	15.75	>200	>200	0.86	8.63

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHIV6248 Rev05
Effective Date: 06/15/2021