



HIV Seroconversion Panel Donor No. 67485
Part Number: HIV9021

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	CHIRON HIV-1 bDNA TEST DATE: 10/26/1999	ABBOTT HIV-1 p24 Ag TEST DATE: 6/16/1999	COULTER HIV-1 p24 Ag TEST DATE: 6/18/1999	ROCHE ELECSYS HIV p24Ag TEST DATE: 6/10/1999	ABBOTT AxSYM HIV AB 1/2 EIA TEST DATE: 7/14/1999	ABBOTT HIV AB 1/2 EIA rDNA TEST DATE: 5/11/1999	ABBOTT HIV AB 1/2 EIA rDNA 3 rd Generation TEST DATE: 6/14/1999	ABBOTT HIV AB 1/2 EIA rDNA IMx TEST DATE: 12/7/1999
9021-01	04/24/98	<50	0.50	0.241	0.32	0.21	0.112	0.20	0.33
9021-02	04/27/98	<50	0.46	0.241	0.31	0.23	0.121	0.21	0.30
9021-03	05/01/98	<50	0.39	0.241	0.32	0.23	0.138	0.21	0.31
9021-04	05/05/98	<50	0.40	0.361	0.34	0.20	0.224	0.20	0.31
9021-05	05/08/98	<50	0.41	0.361	0.30	0.23	0.147	0.22	0.34
9021-06	05/12/98	<50	0.39	0.361	0.29	0.23	0.155	0.22	0.31
9021-07	05/15/98	<50	0.46	0.482	0.31	0.22	0.129	0.22	0.33
9021-08	05/19/98	<50	0.41	0.482	0.30	0.22	0.147	0.22	0.33
9021-09	05/22/98	<50	0.44	0.361	0.31	0.22	0.310	0.21	0.31
9021-10	05/26/98	<50	0.44	0.241	0.32	0.24	0.181	0.21	0.34
9021-11	05/30/98	<50	0.48	0.241	0.36	0.23	0.233	0.22	0.38
9021-12	06/02/98	<50	0.54	0.482	0.40	0.24	0.483	0.22	0.34
9021-13	06/06/98	599	0.39	0.361	0.34	0.25	0.793	0.22	0.34
9021-14	06/10/98	143379	1.43	1.819	0.52	0.24	0.569	0.21	0.30
9021-15	06/13/98	>500000	5.23	9.277	0.96	0.23	0.138	0.30	0.34
9021-16	06/17/98	>500000	8.15	25.422	1.68	0.49	0.716	0.68	1.55
9021-17	06/20/98	>500000	>24.4	41.084	8.69	3.65	2.345	2.27	9.10

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHIV9021 Rev05
Effective Date: 10/25/2021



HIV Seroconversion Panel Donor No. 67485
Part Number: HIV9021

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	ABBOTT MUREX HIV Ab 1/2 + O TEST DATE: UNK	ORTHO VITROS Eci ANTI-HIV 1/2 TEST DATE: 3/23/2015	ABBOTT MUREX SUDS ANTI-HIV-1 TEST DATE: 7/21/1999	ABBOTT PRISM ANTI-HIV 1/2 TEST DATE: 7/15/1999	BIOMERIEUX ORGANON VIRNOSTIKA HIV-1 UNI-FORM II PLUS 0 TEST DATE: 6/9/1999	BIOMERIEUX ORGANON VIRNOSTIKA HIV MICROELISA SYSTEM TEST DATE: 6/18/1999	BIO-RAD NOVAPATH HIV-1 IMMUNOBLOT TEST DATE: UNK
9021-01	0.466	0.18	0.21	0.24	0.45	0.237	NEGATIVE
9021-02	0.425	0.18	0.21	0.23	0.52	0.237	NEGATIVE
9021-03	0.397	0.18	0.25	0.23	0.41	0.208	NEGATIVE
9021-04	0.394	0.17	0.21	0.22	0.47	0.208	NEGATIVE
9021-05	0.400	0.18	0.24	0.21	0.41	0.208	NEGATIVE
9021-06	0.384	0.17	0.26	0.22	0.41	0.237	NEGATIVE
9021-07	0.416	0.17	0.23	0.23	0.43	0.237	NEGATIVE
9021-08	0.494	0.18	0.24	0.22	0.57	0.237	NEGATIVE
9021-09	0.447	0.17	0.27	0.23	0.44	0.237	NEGATIVE
9021-10	0.528	0.15	0.23	0.22	0.37	0.237	NEGATIVE
9021-11	0.475	0.13	0.22	0.23	0.40	0.208	NEGATIVE
9021-12	0.372	0.16	0.22	0.21	0.38	0.237	NEGATIVE
9021-13	0.350	0.17	0.22	0.32	0.41	0.237	NEGATIVE
9021-14	0.419	0.15	0.24	0.27	0.56	0.237	NEGATIVE
9021-15	0.447	0.21	0.21	0.27	0.44	0.237	NEGATIVE
9021-16	3.125	7.92	0.83	0.88	1.10	0.237	NEGATIVE
9021-17	10.697	35.7	8.73	2.43	3.35	0.297	NEGATIVE

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHIV9021 Rev05
Effective Date: 10/25/2021



HIV Seroconversion Panel Donor No. 67485
Part Number: HIV9021

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	BIOTEST ANTI-HIV 1/2 TEST DATE: 6/9/1999	BIOTEST ANTI-HIV-1 TETRA ELISA TEST DATE: 6/8/1999	DADE BEHRING ENZYGNOST ANTI-HIV 1/2 PLUS TEST DATE: 6/8/1999	GENETICS SYSTEMS HIV 1/2 PEPTIDE EIA TEST DATE: UNK	GENETICS SYSTEMS rLAV EIA TEST DATE: 11/15/1999	ORTHO ANTI-HIV ½ ENHANCED ELISA TEST DATE: 6/21/1999	ROCHE ENZYMUM ANTI-HIV- COMBI TEST DATE: 7/14/1999
9021-01	0.04	0.13	0.12	0.176	0.074	0.04	0.30
9021-02	0.02	0.71	0.13	0.176	0.071	0.03	0.30
9021-03	0.02	0.11	0.13	0.141	0.065	0.03	0.30
9021-04	0.02	0.23	0.12	0.141	0.065	0.06	0.30
9021-05	0.00	0.13	0.11	0.176	0.059	0.06	0.30
9021-06	0.00	0.16	0.11	0.211	0.050	0.05	0.30
9021-07	0.00	0.14	0.12	0.106	0.083	0.05	0.30
9021-08	0.01	0.15	0.11	0.141	0.074	0.04	0.30
9021-09	0.00	0.13	0.12	0.141	0.074	0.05	0.30
9021-10	0.00	0.23	0.13	0.106	0.077	0.04	0.30
9021-11	0.00	0.12	0.11	0.106	0.080	0.06	0.30
9021-12	0.01	0.12	0.11	0.141	0.103	0.04	0.30
9021-13	0.01	0.10	0.11	0.141	0.091	0.05	0.30
9021-14	0.00	0.11	0.11	0.106	0.077	0.04	0.30
9021-15	0.01	0.13	0.14	0.176	0.053	0.195	0.58
9021-16	0.29	2.39	0.44	0.317	0.059	9.75	0.80
9021-17	5.13	4.53	2.57	0.176	0.127	13.91	5.28

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHIV9021 Rev05
Effective Date: 10/25/2021



HIV Seroconversion Panel Donor No. 67485
Part Number: HIV9021

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	SANOI GENSREEN ANTI-HIV 1/2 TEST DATE: 6/21/1999	ABBOTT PRISM HIV Ag/Ab COMBO 4 TH Generation TEST DATE: UNK	ABBOTT MUREX HIV-1 Ab/Ag COMBO 4 TH Generation TEST DATE: UNK	ABBOTT ARCHITECT HIV Ag/Ab COMBO 4 TH Generation TEST DATE: UNK	BIO-RAD BioPlex 2200 HIV Ag-Ab 5 th Generation TEST DATE: UNK	BIO-RAD BioPlex 2200 HIV-1 Ag 5 th Generation TEST DATE: UNK	BIO-RAD BioPlex 2200 HIV-1 Ab 5 th Generation TEST DATE: UNK
9021-01	0.11	0.06	0.274	0.18	0.09	0.09	0.03
9021-02	0.11	0.09	0.234	0.10	0.09	0.09	0.06
9021-03	0.10	0.08	0.228	0.09	0.13	0.13	0.04
9021-04	0.11	0.18	0.244	0.10	0.10	0.10	0.07
9021-05	0.09	0.07	0.274	0.13	0.08	0.08	0.05
9021-06	0.10	0.74	0.249	0.14	0.09	0.09	0.05
9021-07	0.06	0.08	0.259	0.12	0.08	0.08	0.05
9021-08	0.07	0.06	0.315	0.15	0.11	0.11	0.08
9021-09	0.06	0.13	0.223	0.13	0.14	0.14	0.06
9021-10	0.06	0.20	0.462	0.17	0.12	0.12	0.04
9021-11	0.61	0.13	0.386	0.27	0.08	0.08	0.02
9021-12	0.08	0.06	0.249	0.15	0.06	0.06	0.02
9021-13	0.06	0.08	0.269	0.13	0.21	0.21	0.06
9021-14	0.06	1.13	1.853	1.49	6.86	6.86	0.08
9021-15	0.07	6.01	9.721	8.59	44.28	44.28	0.04
9021-16	5.44	11.19	15.827	16.46	89.39	89.39	0.10
9021-17	MAX ABS.	65.98	18.787	110.56	>200	>200	1.26

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHIV9021 Rev05
Effective Date: 10/25/2021