



**HIV Seroconversion Panel Donor No. 67996
Part Number: HIV9025**

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	CHIRON HIV-1 bDNA	ABBOTT HIV-1 p24 Ag	COULTER HIV-1 p24 Ag	ROCHE ELECSYS HIV p24Ag	ABBOTT AxSYM HIV AB ½ EIA	ABBOTT HIV AB ½ EIA rDNA	ABBOTT HIV AB ½ EIA rDNA 3 rd Generation	ABBOTT MUREX HIV Ab ½ + O 3 rd Generation
		TEST DATE: 10/22/1999	TEST DATE: 6/16/1999	TEST DATE: 8/5/1999	TEST DATE: 6/21/1999	TEST DATE: 7/14/1999	TEST DATE: 5/13/1999	TEST DATE: 6/22/1999	TEST DATE: UNK
9025-01	05/23/98	<50	0.38	0.490	0.25	0.24	0.174	0.22	0.372
9025-02	05/30/98	<50	0.46	0.686	0.33	0.27	0.215	0.30	0.344
9025-03	06/06/98	<50	0.41	0.686	0.30	0.27	0.207	0.27	0.313
9025-04	06/15/98	<50	0.43	0.588	0.28	0.24	0.174	0.26	0.309
9025-05	06/20/98	<50	0.49	0.588	0.28	0.25	0.190	0.11	0.322
9025-06	06/29/98	<50	0.44	0.490	0.32	0.25	0.174	0.10	0.372
9025-07	07/20/98	<50	0.45	0.490	0.26	0.22	0.190	0.12	0.394
9025-08	07/22/98	<50	0.38	0.686	0.28	0.25	0.174	0.23	0.466
9025-09	07/27/98	<50	0.43	0.490	0.30	0.27	0.157	0.29	0.350
9025-10	07/30/98	<50	0.43	0.588	0.33	0.29	0.182	0.29	0.309
9025-11	08/16/98	>500000	3.18	3.824	9.84	0.24	0.190	0.21	0.303
9025-12	08/22/98	>500000	>24.4	32.353	174.3	0.85	1.727	1.28	4.491

PANEL MEMBER	ABBOTT HIV AB ½ EIA rDNA IMx	ABBOTT MUREX SUDS ANTI-HIV-1	ABBOTT PRISM ANTI-HIV ½	BIOMERIEUX ORGANON VIRNOSTIKA HIV-1 MICROELISA SYSTEM	BIOMERIEUX ORGANON VIRNOSTIKA HIV UNI-FORM II PLUS 0	BIO-RAD NOVAPATH HIV-1 IMMUNOBLOT	BIOTEST ANTI-HIV ½	BIOTEST ANTI-HIV-1 TETRA ELISA
	TEST DATE: 7/19/1999	TEST DATE: 7/21/1999	TEST DATE: 7/15/1999	TEST DATE: 7/27/1999	TEST DATE: 6/9/1999	TEST DATE: UNK	TEST DATE: 6/9/1999	TEST DATE: 6/8/1999
9025-01	0.28	0.14	0.24	0.449	0.50	NEGATIVE	0.01	0.15
9025-02	0.34	0.13	0.22	0.449	0.53	NEGATIVE	0.00	0.18
9025-03	0.32	0.16	0.26	0.470	0.60	NEGATIVE	0.01	0.17
9025-04	0.31	0.15	0.24	0.449	0.44	NEGATIVE	0.01	0.17
9025-05	0.34	0.15	0.21	0.406	0.42	NEGATIVE	0.02	0.17
9025-06	0.30	0.14	0.23	0.427	0.44	NEGATIVE	0.02	0.16
9025-07	0.33	0.18	0.26	0.449	0.42	NEGATIVE	0.03	0.15
9025-08	0.33	0.18	0.33	0.449	0.46	NEGATIVE	0.02	0.17
9025-09	0.32	0.14	0.40	0.449	0.49	NEGATIVE	0.02	0.15
9025-10	0.31	0.13	0.26	0.491	0.60	NEGATIVE	0.00	0.15
9025-11	0.31	0.14	0.25	0.470	0.51	NEGATIVE	0.02	0.14
9025-12	0.99	0.32	1.01	0.470	1.23	NEGATIVE	0.32	4.38

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti HCV by FDA cleared tests.

This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.

ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525

For Customer Support, please visit www.zeptometrix.com or email custserv@zeptometrix.com



**HIV Seroconversion Panel Donor No. 67996
Part Number: HIV9025**

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DADE BEHRING ENZYGNOST HIV INTEGRAL TEST TEST DATE: 6/14/1999	GENETICS SYSTEMS HIV 1/2 PEPTIDE EIA TEST DATE: 6/10/1999	GENETICS SYSTEMS rLAV EIA TEST DATE: 11/15/1999	ORTHO ANTI-HIV 1/2 ENHANCED ELISA TEST DATE: 6/21/1999	ROCHE ENZYMUM ANTI-HIV-COMBI TEST DATE: 7/14/1999	SANOFI GENSCREEN ANTI-HIV 1/2 TEST DATE: 6/21/1999	ABBOTT PRISIM HIV Ag/Ab COMBO 4 TH Generation TEST DATE: UNK	ABBOTT MUREX HIV-1 Ab/Ag COMBO 4 TH Generation TEST DATE: UNK
9025-01	0.04	0.200	NOT TESTED	0.04	0.09	0.07	0.06	0.325
9025-02	0.04	0.200	NOT TESTED	0.05	0.10	0.08	0.24	0.340
9025-03	0.04	0.143	NOT TESTED	0.05	0.10	0.07	0.06	0.234
9025-04	0.04	0.143	NOT TESTED	0.05	0.09	0.07	0.07	0.223
9025-05	0.03	0.200	NOT TESTED	0.08	0.10	0.06	0.11	0.259
9025-06	0.07	0.257	NOT TESTED	0.09	0.09	0.10	0.07	0.228
9025-07	0.05	0.200	NOT TESTED	0.06	0.09	0.09	0.08	0.244
9025-08	0.04	0.171	NOT TESTED	0.05	0.09	0.07	NOT TESTED	0.244
9025-09	0.04	0.229	NOT TESTED	0.05	0.10	0.06	0.08	0.244
9025-10	0.02	0.229	NOT TESTED	0.07	0.10	0.07	0.06	0.239
9025-11	0.18	0.257	0.062	0.04	0.48	0.08	4.86	6.218
9025-12	2.86	0.200	0.065	10.93	7.16	2.17	65.57	17.726

PANEL MEMBER	ABBOTT ARCHITECT HIV Ag/Ab COMBO 4 TH Generation TEST DATE: UNK	BIO-RAD BioPlex 2200 HIV Ag-Ab 5 TH Generation TEST DATE: UNK	BIO-RAD BioPlex 2200 HIV-1 Ag 5 TH Generation TEST DATE: UNK	BIO-RAD BioPlex 2200 HIV-1 Ab 5 TH Generation TEST DATE: UNK	ORTHO VITROS ECi ANTI-HIV 1/2 TEST DATE: 8/30/2018
9025-01	0.22	0.11	0.11	0.02	0.12
9025-02	0.21	0.06	0.03	0.04	0.13
9025-03	0.23	0.05	0.05	0.02	0.13
9025-04	0.19	0.07	0.07	0.03	0.13
9025-05	0.22	0.17	0.17	0.05	0.13
9025-06	0.20	0.07	0.06	0.05	0.14
9025-07	0.22	0.13	0.10	0.09	0.14
9025-08	0.18	0.09	0.09	0.02	0.13
9025-09	0.22	0.11	0.11	0.05	0.12
9025-10	0.23	0.14	0.14	0.10	0.12
9025-11	11.00	27.62	27.62	0.03	0.13
9025-12	198.76	>200	>200	0.13	21.4

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti HCV by FDA cleared tests.

This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525

For Customer Support, please visit www.zeptometrix.com or email custserv@zeptometrix.com