

ZeptoMetrix®

HIV Seroconversion Panel Donor No. 62216 Part Number: HIV9075

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	GEN- PROBE PROCLEIX HIV-1 HCV ASSAY	ROCHE COBAS AMPLICOR HIV-1 MONITOR	ABBOTT HIV-1 p24 Ag	COULTER HIV-1 p24 Ag	ZEPTOMETRIX HIV p24 Ag (pg/mL)	ABBOTT HIV Ab ½ EIA rDNA 3 rd Generation	ABBOTT MUREX HIV Ab ½ + O 3 rd Generation
		TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK
9075-01	08/22/96	NEG	NOT DETECTED	NOT TESTED	0.37	<3.0	0.144	0.424
9075-02	08/24/96	NEG	NOT DETECTED	NOT TESTED	0.247	<3.0	0.153	0.341
9075-03	09/13/96	POS	>750,000	2.30	5.222	22.2	0.161	0.359
9075-04	09/23/96	POS	>750,000	3.33	1.889	30.2	16.949	10.430
9075-05	09/25/96	POS	>750,000	NOT TESTED	1.667	69.7	16.949	10.604

PANEL MEMBER	ORTHO VITROS Eci Anti-HIV 1/2	BIOMERIEUX VIRNOSTIKA ANTI HIV-1	BIO-RAD ANTI HIV-1 rLAV	GENETICS SYSTEMS HIV ½ + O	MEDMIRA ANTI-HIV 1 RAPID TEST	TRINITY BIO-TECH UNI GOLD RECOMBIGEN	BIO-RAD MULTISPOT HIV 1 RAPID TEST	BIO-RAD MULTISPOT HIV 2 RAPID TEST
	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK
9075-01	0.14	0.455	0.041	0.102	NEG	NEG	NEG	NEG
9075-02	0.14	0.370	0.044	0.239	NEG	NEG	NEG	NEG
9075-03	0.18	0.415	0.041	0.167	NEG	NEG	NEG	NEG
9075-04	38.8	0.944	3.764	10.239	POS	POS	POS	NEG
9075-05	33.5	1.086	4.159	10.239	POS	POS	POS	NEG

PANEL MEMBER	ORAQUICK ADVANCE HIV ½ RAPID Ab	BIO-RAD HIV-1 WESTERN BLOT	ABBOTT PRISM HIV Ag/Ab COMBO 4 th Generation	ABBOTT MUREX HIV-1 Ab/Ag COMBO 4 th Generation	ABBOTT HIV Ag/Ab COMBO ARCHITECT 4 th Generation	BIO-RAD BioPlex 2200 HIV Ag-Ab 5 th Generation	BIO-RAD BioPlex 2200 HIV-1 Ag 5 th Generation	BIO-RAD BioPlex 2200 HIV-1 Ab 5 th Generation
	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK
9075-01	NEG	NEG	0.24	0.324	0.063	0.12	0.12	0.03
9075-02	NEG	NEG	0.09	0.275	0.071	0.06	0.04	0.05
9075-03	NEG	NEG	3.42	5.049	7.27	28.08	28.08	0.06
9075-04	POS	gp160, p24+/-, p18 +/-	10.28	16.887	31.86	34.38	34.38	27.61
9075-05	POS	gp160, p55+/-, p40 +/-, p24, p18 +/-	17.14	17.162	51.01	33.38	5.02	33.38

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHIV9075 Rev05
Effective Date: 10/26/2021