

ZeptoMetrix®

HIV Seroconversion Panel Donor No. 75062 Part Number: HIV9079

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	GEN-PROBE PROCLEIX HIV-1 /HCV ASSAY TEST DATE: UNK	ROCHE COBAS AMPLICOR HIV-1 MONITOR TEST DATE: UNK	ZEPTOMETRIX HIV p24 Ag (pg/mL) TEST DATE: UNK	ABBOTT HIV AB 1/2 EIA rDNA TEST DATE: UNK	ABBOTT MUREX HIV Ab ½ + O 3 RD Generation TEST DATE: UNK	BIOMERIEUX VIRONOSTIKA ANTI-HIV-1 TEST DATE: UNK	BIO-RAD ANTI-HIV-1 rLAV TEST DATE: UNK	GENETICS SYSTEMS HIV 1/2 + O TEST DATE: UNK
9079-01	11/01/99	NEGATIVE	NEGATIVE	<3.0	0.171	0.483	0.298	0.055	0.648
9079-02	11/03/99	NEGATIVE	NEGATIVE	<3.0	0.117	0.409	0.303	0.063	0.133
9079-03	11/10/99	NEGATIVE	NEGATIVE	<3.0	0.126	0.359	0.506	0.048	2.116
9079-04	11/19/99	NEGATIVE	NEGATIVE	<3.0	0.153	0.337	0.325	0.055	0.580
9079-05	11/21/99	NEGATIVE	NEGATIVE	<3.0	0.117	0.347	0.346	0.055	0.177
9079-06	11/29/99	NEGATIVE	NEGATIVE	<3.0	0.135	0.415	0.337	0.048	0.222
9079-07	12/04/99	NEGATIVE	NEGATIVE	<3.0	0.090	0.449	0.314	0.092	0.102
9079-08	12/06/99	POSITIVE	NEGATIVE	<3.0	0.144	0.486	0.333	0.048	0.352
9079-09	12/11/99	POSITIVE	271000	21.9	0.108	0.563	0.270	0.044	0.287
9079-10	12/13/99	POSITIVE	>750000	99.7	0.135	0.402	0.317	0.059	0.119
9079-11	12/18/99	POSITIVE	>750000	>125	2.839	9.226	0.348	0.170	1.065
9079-12	12/20/99	POSITIVE	>750000	>125	10.763	10.375	0.337	1.044	2.679
9079-13	12/26/99	POSITIVE	211000	10.8	4.169	10.452	0.418	3.572	9.689
9079-14	12/28/99	POSITIVE	46300	4.6	3.958	10.492	0.416	3.362	9.949
9079-15	01/02/00	POSITIVE	11900	<3.0	6.331	10.814	0.488	3.686	10.239
9079-16	01/04/00	POSITIVE	11500	<3.0	11.678	11.050	0.474	3.336	10.239
9079-17	01/09/00	POSITIVE	21700	<3.0	16.949	10.786	0.679	3.978	10.239
9079-18	01/11/00	POSITIVE	20400	<3.0	16.949	11.012	0.665	3.347	10.239
9079-19	01/16/00	POSITIVE	18800	<3.0	16.949	10.842	1.691	4.653	10.239
9079-20	01/18/00	POSITIVE	26500	<3.0	16.949	10.929	1.254	5.077	10.239
9079-21	01/23/00	POSITIVE	18700	<3.0	16.949	10.768	1.528	5.224	10.239
9079-22	01/25/00	POSITIVE	29600	<3.0	16.949	10.820	1.684	5.089	10.239
9079-23	01/30/00	POSITIVE	23800	<3.0	16.949	10.910	1.907	5.424	10.239
9079-24	02/01/00	POSITIVE	21100	<3.0	16.949	10.854	1.887	5.465	10.239
9079-25	02/06/00	POSITIVE	15400	<3.0	16.949	10.830	2.702	6.483	10.239

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHIV9079 Rev07
Effective Date: 06/15/2021

ZeptoMetrix®

HIV Seroconversion Panel Donor No. 75062 Part Number: HIV9079

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	MEDMIRA ANTI-HIV-1 RAPID TEST	ORAQUICK ADVANCE HIV ½ RAPID Ab	TRINITY BIOTECH UNIGOLD RECOMBIGNEN HIV-1	BIO-RAD MULTISPOT HIV 1 RAPID TEST	BIO-RAD MULTISPOT HIV 2 RAPID TEST	BIO-RAD HIV-1 WESTERN BLOT
	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK
9079-01	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	NOT TESTED
9079-02	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	NOT TESTED
9079-03	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	NOT TESTED
9079-04	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	NOT TESTED
9079-05	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	NOT TESTED
9079-06	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	NOT TESTED
9079-07	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	NOT TESTED
9079-08	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	NOT TESTED
9079-09	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE
9079-10	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE
9079-11	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE
9079-12	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	P24+/-
9079-13	POSITIVE	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	GP160+/-,P55+/-,P40+/-,P24
9079-14	POSITIVE	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	GP160+/-,P55+/-,P40,P24
9079-15	POSITIVE	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	GP160,P55,P40,P24
9079-16	POSITIVE	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	GP160,P55,P40,P24
9079-17	POSITIVE	NEGATIVE	NEGATIVE	POSITIVE	NEGATIVE	GP160,P55,P40,P24
9079-18	POSITIVE	NEGATIVE	NEGATIVE	POSITIVE	NEGATIVE	GP160,P55,P40,P24
9079-19	POSITIVE	POSITIVE	POSITIVE	POSITIVE	NEGATIVE	GP160,P65+/-,P55,P40,P31+/-,P24,P18+/-
9079-20	POSITIVE	POSITIVE	POSITIVE	POSITIVE	NEGATIVE	GP160,P65+/-,P55,P40,P31+/-,P24,P18+/-
9079-21	POSITIVE	POSITIVE	POSITIVE	POSITIVE	NEGATIVE	GP160,P65+/-,P55,P40,P31+/-,P24,P18+/-
9079-22	POSITIVE	POSITIVE	POSITIVE	POSITIVE	NEGATIVE	GP160,GP120+/-,P65+/-,P55,P40,P31+/-,P24,P18+/-
9079-23	POSITIVE	POSITIVE	POSITIVE	POSITIVE	NEGATIVE	GP160,GP120+/-,P65,P55,GP41+/-,P40,P31,P24,P18
9079-24	POSITIVE	POSITIVE	POSITIVE	POSITIVE	NEGATIVE	GP160,GP120+/-,P65,P55,GP41+/-,P40,P31,P24,P18
9079-25	POSITIVE	POSITIVE	POSITIVE	POSITIVE	NEGATIVE	GP160,GP120+/-,P65,P55,GP41+/-,P40,P31,P24,P18

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHIV9079 Rev07
Effective Date: 06/15/2021

ZeptoMetrix®

HIV Seroconversion Panel Donor No. 75062 Part Number: HIV9079

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	ABBOTT PRISM HIV Ag/Ab COMBO 4 th Generation TEST DATE: UNK	ABBOTT MUREX MICROPLATE HIV Ag/Ab COMBO 4 TH Generation TEST DATE: UNK	ABBOTT HIV Ag/Ab COMBO ARCHITECT 4 TH Generation TEST DATE: UNK	BIO-RAD BioPlex 2200 HIV Ag-Ab 5 th Generation TEST DATE: UNK	BIO-RAD BioPlex 2200 HIV-1 Ag 5 th Generation TEST DATE: UNK	BIO-RAD BioPlex 2200 HIV-1 Ab 5 th Generation TEST DATE: UNK	ORTHO VITROS ECI ANTI-HIV 1/2 TEST DATE: 8/4/2015
9079-01	0.09	0.260	0.06	0.07	0.07	0.01	0.14
9079-02	0.06	0.235	0.13	0.07	0.07	0.02	0.12
9079-03	0.06	0.225	0.09	0.08	0.08	0.00	0.12
9079-04	0.06	0.235	0.16	0.12	0.12	0.02	0.12
9079-05	0.06	0.250	0.06	0.10	0.10	0.06	0.12
9079-06	0.05	0.260	0.07	0.08	0.08	0.01	0.13
9079-07	0.06	0.245	0.13	0.06	0.06	0.03	0.12
9079-08	0.06	0.255	0.07	0.08	0.07	0.02	0.12
9079-09	3.07	2.833	3.58	11.24	11.24	0.01	0.07
9079-10	13.27	12.892	23.85	80.13	80.13	0.01	0.16
9079-11	31.37	16.549	43.06	160.30	160.30	0.06	29.1
9079-12	50.03	16.363	68.68	>200	>200	0.19	47.1
9079-13	25.39	9.225	34.81	4.75	4.75	4.49	26.9
9079-14	3.61	10.270	3.78	5.11	3.10	5.11	26.2
9079-15	3.61	16.407	4.13	3.95	0.20	3.95	21.5
9079-16	6.01	16.863	8.38	3.33	0.17	3.33	19.6
9079-17	6.61	16.838	12.83	2.11	0.07	2.11	23.5
9079-18	6.44	16.598	13.51	1.74	0.12	1.74	24.6
9079-19	9.85	16.833	19.63	2.94	0.02	2.94	QNS
9079-20	9.33	16.730	20.99	3.72	0.07	3.72	37.2
9079-21	9.94	16.863	24.17	4.42	0.02	4.42	45.6
9079-22	9.51	16.917	24.23	4.28	0.10	4.28	42.2
9079-23	9.74	17.059	25.15	5.82	0.06	5.82	45.7
9079-24	10.17	17.627	27.52	6.22	0.04	6.22	48
9079-25	10.10	17.083	29.39	7.63	0.11	7.63	51.7

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHIV9079 Rev07
Effective Date: 06/15/2021