

ZeptoMetrix®

HIV Seroconversion Panel Donor No. 77840 Part Number: HIV9082

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ROCHE COBAS AMPLICOR HIV-1 MONITOR TEST DATE: UNK	ZEPTOMETRIX HIV p24 Ag (pg/mL) TEST DATE: UNK	ORTHO VITROS Eci ANTI-HIV 1/2 TEST DATE: 8/6/2015	ABBOTT HIV AB 1/2 EIA rDNA TEST DATE: UNK	GENETICS SYSTEMS HIV 1/2 + O TEST DATE: UNK	MEDMIRA ANTI-HIV-1 RAPID TEST TEST DATE: UNK	ORAQUICK ADVANCE HIV 1/2 RAPID Ab TEST DATE: UNK
9082-01	01/30/00	NEGATIVE	0.8	0.13	0.3	0.1	NOT TESTED	NOT TESTED
9082-02	02/03/00	NEGATIVE	0.94	0.14	0.1	0.1	NOT TESTED	NOT TESTED
9082-03	02/12/00	52800	1.94	0.16	0.1	0.1	NEGATIVE	NEGATIVE
9082-04	02/19/00	604000	2.24	18.2	3.6	0.5	NEGATIVE	NEGATIVE

PANEL MEMBER	TRINITY BIO-TECH UNI GOLD RECOMBIGEN HIV-1 TEST DATE: UNK	BIO-RAD MULTISPOT HIV 1 RAPID TEST TEST DATE: UNK	CHEMBIO STAT PAK RAPID HIV ½ TEST DATE: UNK	CHEMBIO SURE CHECK RAPID HIV ½ TEST DATE: UNK	CAMBRIDGE BIOTECH HIV-1 WESTERN BLOT TEST DATE: UNK	BIO-RAD BioPlex 2200 HIV Ag-Ab 5 th Generation TEST DATE: UNK	BIO-RAD BioPlex 2200 HIV-1 Ag 5 th Generation TEST DATE: UNK	BIO-RAD BioPlex 2200 HIV-1 Ab 5 th Generation TEST DATE: UNK
9082-01	NOT TESTED	NOT TESTED	NOT TESTED	NOT TESTED	NOT TESTED	0.06	0.06	0.01
9082-02	NOT TESTED	NOT TESTED	NOT TESTED	NOT TESTED	NOT TESTED	0.12	0.12	0.01
9082-03	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	6.77	6.77	0.01
9082-04	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	>200	>200	0.64

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHIV9082 Rev05
Effective Date: 10/26/2021