

ZeptoMetrix®

HIV Seroconversion Panel Donor No. 77600 Part Number: HIV9084

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ORTHO VITROS ECi ANTI-HIV 1/2 TEST DATE: UNK	CHIRON HIV-1 RNA bDNA TEST DATE: 10/18/2000	ROCHE COBAS AMPLICOR HIV-1 MONITOR TEST DATE: UNK	BIO-RAD BioPlex 2200 HIV Ag-Ab 5 th Generation TEST DATE: UNK	BIO-RAD BioPlex 2200 HIV-1 Ag 5 th Generation TEST DATE: UNK	BIO-RAD BioPlex 2200 HIV-1 Ab 5 th Generation TEST DATE: UNK	ZEPTOMETRIX HIV p24 Ag (pg/mL) TEST DATE: 8/23/2007	ABBOTT HIV AB 1/2 EIA rDNA TEST DATE: UNK
9084-01	02/07/00	0.12	<50	NEGATIVE	0.13	0.13	0.07	0.5	0.15
9084-02	02/10/00	0.11	<50	NEGATIVE	0.13	0.13	0.08	0.4	0.14
9084-03	02/17/00	0.12	1077	1290	0.20	0.20	0.10	0.6	0.14
9084-04	03/27/00	30.6	29097	85400	32.31	0.80	32.31	0.6	14.3

PANEL MEMBER	GENETICS SYSTEMS rLAV EIA TEST DATE: UNK	MEDMIRA ANTI-HIV-1 RAPID TEST TEST DATE: UNK	ORAQUICK ADVANCE HIV 1/2 RAPID Ab TEST DATE: UNK	TRINITY BIO-TECH UNI GOLD RECOMBIGEN HIV-1 TEST DATE: UNK	BIO-RAD MULTISPOT HIV 1 RAPID TEST TEST DATE: UNK	CHEMBIO STAT PAK RAPID HIV 1/2 TEST DATE: UNK	CHEMBIO SURE CHECK RAPID HIV 1/2 TEST DATE: UNK	CAMBRIDGE BIOTECH HIV-1 WESTERN BLOT TEST DATE: UNK
9084-01	0.1	NOT TESTED	NOT TESTED	NOT TESTED	NOT TESTED	NOT TESTED	NOT TESTED	NOT TESTED
9084-02	0.1	NOT TESTED	NOT TESTED	NOT TESTED	NOT TESTED	NOT TESTED	NOT TESTED	NOT TESTED
9084-03	0.1	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE
9084-04	5.5	POSITIVE	POSITIVE	POSITIVE	POSITIVE	POSITIVE	POSITIVE	POSITIVE

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHIV9084 Rev06
Effective Date: 10/26/2021